VOCO

Admira® Bond

Instructions for use

Product description:

Admira Bond is an ORMOCER®-based light-curing one-component dentine-enamel bond. **Admira Bond** contains specially developed adhesive ORMOCER®s as well as additive aliphatic and aromatic dimethacrylates. ORMOCER®s as a material class are protected by patent. **Admira Bond** cures under halogen light/LED light (blue light).

Indications:

Admira Bond is suited for direct restorations of all filling classes using light-curing restorative material.

Directions:

Cavity preparation:

In general, minimal preparation according to the rules of the adhesive filling technique should be preferred in order to conserve as much healthy tooth substance as possible (rounded off, box-shaped cavities).

Cleaning/Drying:

Cleaning and drying of the cavity is carried out with special preparations or hydrogen peroxide solution. Due to the subsequent application of light-curing restorative material, drying of the working field is required. Prevent any contamination of the cavity with blood or saliva.

Lining:

A calcium hydroxide lining should be applied in close proximity to the pulp. A lining is recommended for very deep cavities. Due to their fluoride release, glass ionomer cements are particularly suitable for this purpose.

Beveling/etching:

Use **Admira Bond** in the "total-etch technique". Bevel the enamel and then etch the entire cavity, starting from the enamel margin (enamel approx. 20 - 30 s, dentine for a maximum of 15 s with **Vococid**). In case of fluoridated enamel or enamel with a low density of prisms, prolong the etching time **on the enamel only** accordingly (up to 90 s). Then remove etchant, rinse for 20 s and remove excess moisture with the air syringe. The dentine surface should remain slightly moist (not too dry) and the enamel surface should have a chalky white appearance.

Application of Admira Bond:

Dispense **Admira Bond** onto a **Single Tim** or a disposable brush. As an alternative, dispense **Admira Bond** onto a mixing pallet. Apply **Admira Bond** evenly on all prepared enamel and dentine surfaces using a **Single Tim** or a disposable brush and allow to act for 30 s. Then dispense **Admira Bond** with a faint airjet and polymerise with halogen light/LED light for 20 s. Conventional polymerisation devices are suited for light-curing this material. The light output should be a minimum of 500 mW/cm² for halogen polymerisation devices and 300 mW/cm² for LED devices. Do not remove or contaminate the inhibition layer since it is important for the adhesion to the restorative material.

Filling:

Use the restorative material according to the manufacturer's instructions.

Information, precautionary measures:

- There are no known side effects. However, hypersensitive persons may have a reaction.
- **Admira Bond** contains ORMOCER®s, Bis-GMA, HEMA, BHT, acetone and organic acids. **Admira Bond** should not be used in case of known hypersensitivities (allergies) to any of these ingredients. Avoid contact with the oral mucosa. In case of contact of **Admira Bond** with the eyes, rinse immediately with plenty of water and seek medical advice.
- Materials containing thymol, eugenol or oil of cloves interfere with the curing of restorative composites and thus should not be used in combination with composite fillings.

Storage:

Tightly close **Admira Bond** bottle immediately after use. Store at temperatures of 4 °C to 23 °C (storage at 5 °C - 8 °C prolongs the shelflife of the product). Do not expose **Admira Bond** to direct sunlight or the chair light. Do not use after expiry date.

Our preparations have been developed for use in dentistry. As far as the application of the products delivered by us is concerned, our verbal and/or written information has been given to the best of our knowledge and without obligation. Our information and/or advice do not relieve you from examining the materials delivered by us as to their suitability for the intended purposes of application. As the application of our preparations is beyond our control, the user is fully responsible for the application. Of course, we guarantee the quality of our preparations in accordance with the existing standards and corresponding to the conditions as stipulated in our general terms of sale and delivery.

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MD EU Medical device

Product description:

Vococid is an etchant which can be used in the enamel etching technique or total-etch technique. The etching gel contains 35 % phosphoric acid, the etching liquid contains 36,1 % phosphoric acid.

Indications:

- Vococid** is intended to etch only the part of the enamel or the dentine (in total-etch technique), which will subsequently be covered by a composite material:
- for composite fillings
 - for adhesive luting of Maryland bridges, inlays, brackets and temporary splinting
 - for sealing as caries prophylaxis
 - for linings to create a permanent bond (according to manufacturer's instructions)

Contraindications:

Vococid is contraindicated in patients with a known sensitivity or allergy to device chemical materials.

Patient target group:

Vococid is suitable for use in all patients without any age or gender restrictions.

Performance features:

The product's performance features satisfy the requirements of the intended use and the relevant product standards.

User:

Vococid should only be used by a professionally trained dental practitioner.

Use:

Vococid is both simple to use and easily visible on the enamel due to the blue colouring and consistency of the product:

With **Vococid** gel an exact application is possible (e. g. for etching the enamel margins) and the liquid is suitable for treatment of larger surfaces.

Before using the gel for the first time and also after long storage times, please apply just a little bit of the gel on a mixing pad to get a feeling for the consistency and the application characteristics of the material.

Without total-etching (only etching of enamel):

Application of the gel: Apply **Vococid** directly from the cannula onto the tooth. Application of the liquid: Coat the surface of the enamel with one or two applications of **Vococid** on a **Pele Tim**.

The enamel surface must be completely moist. If necessary, apply extra etching liquid/gel.

Vococid should be allowed to act for the recommended period of time as shown below:

approx. 20 s for regular enamel and for enamel of adolescents; approx. 30 s for devitalized teeth, deeply fluoridated enamel or enamel without prisms (e.g. deciduous teeth).

After application rinse thoroughly with water, spraying each etched tooth for at least 15 - 30 s. Afterwards the patient should not close his/her mouth. Dry the etched surfaces thoroughly using an air blower.

Absolutely avoid contamination with saliva. In case of contamination, repeat the etching process (10 s). Check the etched areas (chalky-white).

Total-etch-technique with dentine and enamel bonding materials (etching of enamel and dentine):

Etch entire cavity (enamel approx. 20 - 30 s, dentine max. 15 s). Remove **Vococid** etching gel/liquid by suction, rinse and remove excess moisture with an air-blower.

Instructions for filling an empty 2 ml syringe:

- Wear protective gloves and safety glasses!
- Remove the protection caps from the 60 ml supply syringe and the empty 2 ml syringe.
- Ensure that the plunger of the empty 2 ml syringe is pressed down completely.
- Attach the 2 ml syringe to the 60 ml syringe and join them firmly by turning the 2 ml syringe clockwise (Caution: Do not overtighten the thread!).
- Decant the etching gel from the 60 ml syringe into the 2 ml syringe by slowly pulling the plunger of the 2 ml syringe (do not decant etching gel from the 2 ml syringe back into the 60 ml syringe!).
- Detach the syringes from each other by slowly turning the 2 ml syringe anticlockwise and close both syringes with their respective protection caps.
- To ensure batch traceability write the batch number and expiry date of the 60 ml syringe in use on one of the labels provided and attach it to the filled 2 ml syringe.
- To use the 2 ml syringe remove the protection cap, attach the application cannula and proceed in accordance with the above mentioned instructions for use.

Please note: 2 ml syringes may only be filled once, do not refill a second time! The application cannulae, too, are disposable devices. Syringes and application cannulae cannot be sterilised.

Warnings, precautionary measures:

- **Caution! Vococid** contains phosphoric acid!
- Wear safety glasses and protective gloves!
- Avoid contact with eyes, skin and mucous membranes. In the event of accidental contact, rinse with plenty of water. Do not swallow!
- In case of contact with eyes, rinse the eyes with water for 15 minutes and seek medical advice.
- Avoid contact with gingiva, lining materials containing calcium hydroxide and neighbouring teeth.
- If total etching is not used, cover the dentine before etching, if necessary. It is recommended to cover adjacent teeth and etched enamel surfaces, which are not covered with composite material, once or twice with a fluoride varnish for remineralisation.
- Our information and/or advice do not relieve you of the obligation of checking that the products supplied by us are suitable for their intended purpose.

Storage instructions and application method:

Storage at 4°C-23°C. Tightly close after use. Only give completely emptied packages to recycling centers. Do not use after the expiry date.

Disposal:

Dispose of the product in accordance with local regulations.

Vococid residues can be disposed of with waste water, when highly diluted with water or neutralised.

Reporting obligation:

Serious events such as death, temporary or permanent serious deterioration of a patient's, user's or other person's health condition and a serious risk to public health that arise or could have arisen in association with the use of **Vococid** must be reported to VOCO GmbH and the responsible authority.

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VC 60 BB2475 E1 0117 99 © by VOCO
ORMOCER® is a registered trademark of Fraunhofer Gesellschaft zur Förderung der angewandten Forschung e.V., München, Germany.