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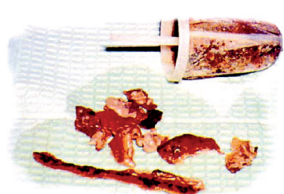
*A correct maintenance will increase the efficiency of the aspirator and avoid most operating problems. During dental surgery, saliva, blood and mucous are aspirated; putrefiable material that can quickly decompose and sticks to the internal walls of the aspirator and suction ducts.*

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*If a strategy is not implemented to protect a new aspirator in the first days of its use using the appropriate products, implementing well-organised methods to fight the stratifications of biofilm and constantly control bacterial growth, a patina will form which will firstly block suction and later completely seal up various points in the aspirator's pipings. At this point cleansing once in a while with aggressive products will not suffice to restore the aspirator to its original condition: strong products cause more damage to the aspirator than the bacterial formations themselves.*

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*If used properly and regularly, ensure good cleansing and disinfection of the aspirator's internal parts and pipings. We recommend **Puli-Jet Plus New** with anti-scale agent and antifoam disinfectant tablets. The product - **Puli-Jet Plus New** with anti-scale agent - if correctly diluted, can remain in contact with the aspirator, inside it and its pipings, as long as possible without damaging the treated surfaces. This is why we recommend it for use especially for pre-holiday and overnight disinfection and advise you not to rinse after disinfection till before using the aspirator.*

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*We recommend placing Disinfectant Antifoam Tablets in the aspiration unit's filter after cleansing and disinfection. Blood and mucous, diluted in the turbine's water, or in the syringe and shaken by a flow of pressurized air, can generate a large quantity of foam. The foam then carries bacteria everywhere, in a dark and damp environment, perfect for their proliferation.*

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*Disinfection carried out at morning, a few minutes before starting work, does not have the same effect as overnight disinfection. What was aspirated the day before and the biological transformation which occurred during the night are not cancelled out by disinfection treatment the next day. Normally, treatment with **Puli-Jet Plus New** is advised half-way through the day and at night. Also, it is advisable to carry out a disinfection treatment straight after treating a patient who is known to have an infectious disease. The concentrated liquid should be diluted according to the instructions on the container and the solution must be aspirated using the Pulse-Cleaner. In fact, the pulsating bucket creates the turbulence necessary to remove dirt so that the cleanser/disinfectant can destroy it. Using **Puli-Jet Plus New** with anti-scale agent dissolves blood and mucous and removes scale from the aspirator walls and pipings which will then be clean.*

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*During the entire working day, the aspirated liquid passes through the filter containing the disinfectant antifoam tablets and collects a few particles of disinfectant. The rinses described will keep internal surfaces clean and disinfected and overnight, the solution aspirated during the last treatment will complete the recommended antibacterial protection strategy.*