



Bisi-White Office

Professional In-Office Teeth Whitening System

**Brilliant Results.
Simple Application.
Professional Whitening**



Complete Set



No lamp



Low investment



Made in Germany



Why Bisi-White Office?

Effective Whitening

Bisi-White provides powerful whitening for discolored or devitalized teeth, delivering long-lasting results.

Fast 30-Minute Whitening

The treatment is highly effective, using 3 cycles of 10 minutes for a total of just 30 minutes.

Optimal Formula

With a neutral pH value and ideal consistency, Bisi-White is gentle on teeth while still being highly effective.



No lamp or laser

The whitening process works entirely without light or heat. A costly investment in lamps or lasers is completely eliminated.



How to use

- 1. Apply Gingiva Protector**
and polymerise for 20 to 40 sec.
- 2. Apply Bisi-White gel**
remove it after 10 min. and repeat max. 2 more times.
- 3. Apply After-Bleaching-Care**
to remineralise teeth and protect against sensitivities.



For more
information
scan QR-Code



bisico[®]

**Set
Content:**

1,5g Bisi-Gingiva Protector incl. 3 Appli Tips
5g Bisi-White Office 35% incl. 3 Mixing-Tips
2ml Bisi-After Care