

PREMIUM WHITENING

CAVEX BITE&WHITE ALIGNER WHITENING PEN

-

The solution for whitening
teeth while your patient
wears an aligner.



YOUR
IMPRESSION
IS OUR
SPECIALTY

CAVEX

PREMIUM WHITENING

CAVEX BITE&WHITE ALIGNER WHITENING PEN

Is your patient about to start an aligner treatment?

Combine it with the Aligner Whitening Pen.

Your patient can correct the alignment of their teeth while whitening them at the same time.

Two treatments in one? Perfect.

How does it work?

The magic of the Aligner Whitening Pen lies in its advanced HPS technology. This technology uses pure hydrogen peroxide in a water-based gel, which is safe and effective for use with aligners.

When the gel comes into contact with teeth and saliva, the hydrogen peroxide breaks down into oxygen, water, and radicals. These free radicals eliminate discoloration by breaking the chemical bonds that cause it.

The result? Whiter teeth, day by day.

Advantages at a glance

- **Easy to use:** Thanks to the pen-shaped design, your patient can precisely apply the gel to the teeth to be whitened.
- **Tooth-friendly:** The gel contains less than 3% hydrogen peroxide, making it safe for regular use.
- **Antibacterial:** The hydrogen peroxide prevents the buildup of plaque and biofilm during aligner wear.
- **Multitasking:** Combine the aligner treatment with a professional whitening treatment.
- **Flexible:** The gel can be used for 20 minutes to up to 2 hours a day, depending on the patient's personal preference.
- **No chair time:** The patient performs the whitening treatment themselves at home.
- **Storage:** The gel can be stored at room temperature.



YOUR
IMPRESSION
IS OUR
SPECIALTY

Fustweg 5 | 2031 CJ Haarlem | The Netherlands | cavex.nl

CAVEX

ZYBW238EN.00