CALASEPT® Plus

Calcium Hydroxide

Calcium hydroxide paste
Kalciumhydroxidpasta
Calciumhydroxid paste
Pâte d'hydroxyde de calcium
Pasta di idrossido di calcio
Kalsiumhydroksidipasta
Calciumhydroxidpasta
Pasta de hidróxido cálcico
Pasta de hidróxido de cálcio
Calciumhydroxidepasta
Wodorotlenek wapnia w pastcie
πάστα στείρου υδροξειδίου του ασβεστίου
Материал стоматологический пломбирочный временный

CE
0413

Made in
Sweden

Rx Only


DIRECTA AB

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DIRECTA INC

64 Barnabas Road, Unit 3
Newtown, CT 06470, United States
203.491.2273

DIRECTA
directadental.com

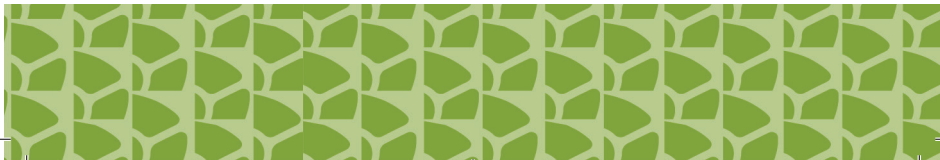106895-2019_04-16

INSTRUCTIONS FOR USE

Calcium hydroxide paste with radiopacity

Indications

1. Temporary root filling material for conventional root treatment.
2. Pulp capping.
3. Perforation and pulp protection.
4. Insulation in deep cavities.

Advantages

- Stimulates formation of dentine bridges.
- Has strong bactericidal effect.
- Stimulates hard tissue formation in roots.
- Quick and simple application of calcium hydroxide.
- Eliminates mixing of powder and liquid.
- Always the ideal consistency.

Contents

Calcium hydroxide 41%, Barium sulphate 5%, Sterile isotonic saline solution, pH 12.4

Directions for use

1. Before application eject a small amount of paste - to equalise pressure in the syringe and to control flow of paste through the Flexi-Tip.
2. For a temporary root filling apply paste gradually and intermittently compress with absorbing material like paper points or cotton pellets.
3. Each syringe can be used several times. To guarantee perfect function the Flexi-Tip should be discarded and replaced with the security cap after use.

All contact between calcium hydroxide paste and air will result in formation of calcium carbonate which will harden after some time.

Additional notes

Calasept is corrosive.

- Skin contact: Wash with plenty of water. Remove splashed clothes and wash underlying skin.
- Eye contact: Immediately rinse with copious amounts of water and seek medical advice.
- Ingestion: Drink water or preferably milk (0,5 liter), and seek medical advice.

Shelf life

Two years from date of manufacture.

Storage

Store at room temperature, do not refrigerate. Recommended storage < 25°C. Avoid exposure to direct sunlight.

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