
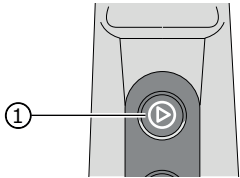
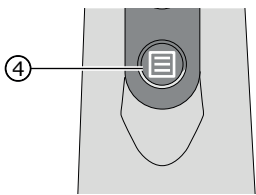
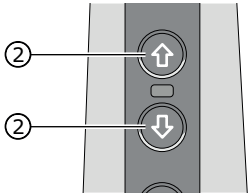
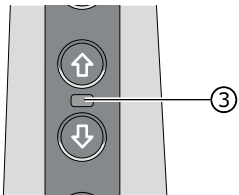






Quick Reference - DIAGNOcam Vision Full HD







| DIAGNOcam Vision Full HD | | |
|--|---|---|
| Installation / Setup |  | ► For software installation and start up note the instructions for use and the installation instructions of the software. |
| Power on | | The camera can be switched on by the following means: <ul style="list-style-type: none"> • System start of the imaging software • Pressing any button on the control panel • Removal of the camera from the table holder or wall mount |
| Capture an image Start / stop a video |  | ► Press the trigger button ① for capturing an image and/or for starting/stopping the video recording. |
| Switch between modes |  | ► To switch between the modes (intraoral, fluorescence and transillumination), press the menu button ④. |
| Navigation of tooth chart |  | ► Press the buttons ② for navigation in the numbered tooth chart (if available in your software). |
| Status LED |  | Status LED ③ indicating the status of the camera: <ul style="list-style-type: none"> • Withe = Camera is active • Blue = Standby mode • Red = Error |
| Power off | | The camera can be switched off by the following means: <ul style="list-style-type: none"> • Placing the camera in the table holder or wall mount • Extended period of inactivity |

Quick Reference - DIAGNOcam Vision Full HD

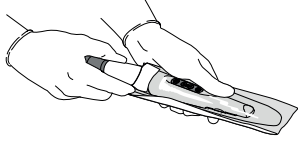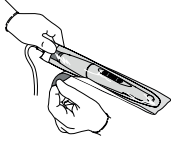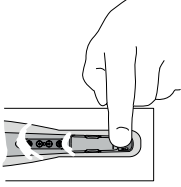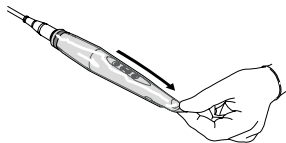

| Using the sheath | | |
|-------------------|--|--|
| Attach the sheath |  | ▶ Slowly apply the sheath over the camera to allow the air to escape. |
| |  | ▶ Remove the outer protective paper. |
| |  | ▶ Tighten the clear view window in front of the exposure window of the camera. ▶ The clear view window of the sheath must be situated exactly over the exposure window of the camera. |
| Remove the sheath |  | ▶ Carefully remove the sheath after each treatment and dispose of it. |
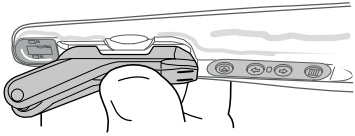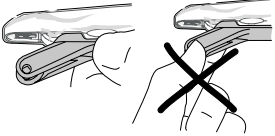
| Snap-in and detach the tip | | |
|----------------------------|--|--|
| Snap-in the tip |  | ▶ Position the tip on one side of the housing and carefully snap-in into the opposite side. ▶ Check on the stable positioning of the tip. |
| Detach the tip |  | ▶ Detach the tip carefully by rotating it on one side, then press off the tip at the predetermined recess and do not touch it by the soft silicone legs. ▶ After every use process the tip in accordance with the instructions for use. |
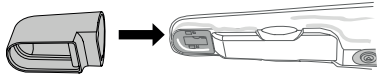
| Attach and detach the fluorescence cover | | |
|--|--|---|
| Attach the fluorescence cover |  | ▶ Position the fluorescence cover on the housing. ▶ Check the stable positioning of the fluorescence cover. |
| Detach the fluorescence cover |  | ▶ Detach the fluorescence cover carefully. ▶ After every use process the fluorescence cover in accordance with the instructions for use. |