



DIAGNOdent® pen 2190

Brief operating instruction

Select probe
storage location:

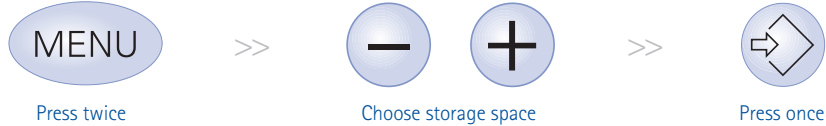


Probe calibration with reference:

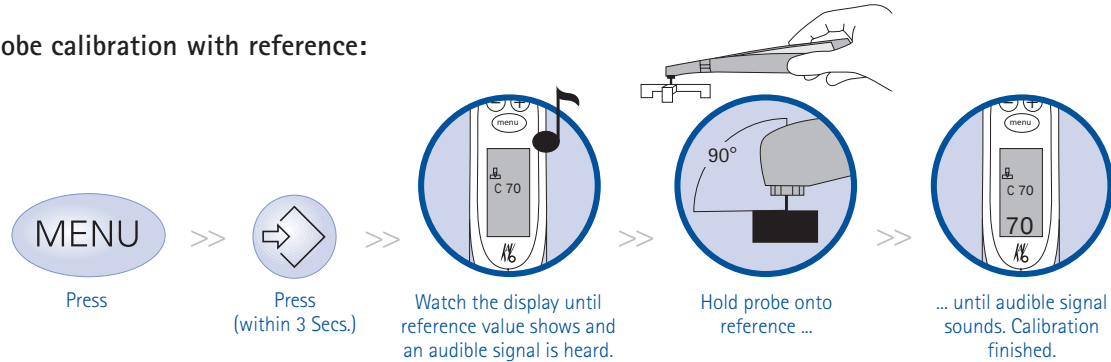


Customise according to patient:

Choose caries free spot – Press switch for 3 Seconds

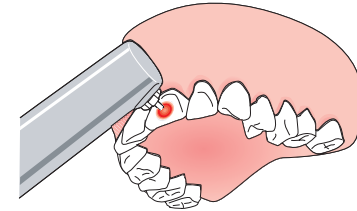

Occlusal-probe

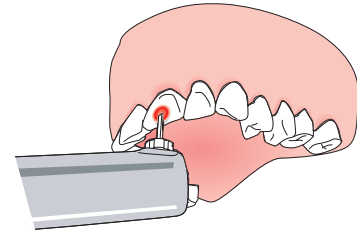

Approx-probe



KaVo. Dental Excellence.

KaVo Dental GmbH · D 88400 Biberach/Riß
Telefon +49 7351 56-0 · Fax +49 7351 56-1488 · www.kavo.com



Those short operating instructions contain only the essential control functions. Before operating for the first time and in the case of operation by persons who are not familiar with handling this medical product, it is essential to read the Comprehensive Operating Instructions included in the delivery.

① Professional tooth cleaning

Removal of all hard and soft plaque with:

SONICflex,
hand instruments,
PROPHYflex with PROPHYpearls,
Polishing handpiece DURAtec
2933

② Examination with DIAGNOdent pen 2190

fissure caries Value: 0 – 12	fissure caries Value: 13 – 24	fissure caries Value: > 25
Healthy tooth substance	Beginning demineralisation	Strong demineralisation
Value: 0 – 7 proximal caries	Value: 8 – 15 proximal caries	Value: > 16 proximal caries
③ Dentists diagnosis		
no findings	monitoring	x-ray, caries bacteria test, saliva test
④ Measures		
standard prophylactic measures	intensive prophylactic measures local antibacterial measures	Minimally invasive treatment
Small risk		High risk
Fluoride toothpaste	fluoridation, KaVo HealOzone, Chlorhexidine	KaVo HealOzone, RONDOflex, SONICflex micro, Composite filling materials and intensive prophylactic