




The Paro probe (Mat. No. 1.004.0370) can be used to detect concretions in pockets up to 9mm deep.

! The probe can break and cause injury.
Do not apply leverage to the probe!

! The probe must be re-setup before each treatment. Please follow the setup instructions for the fissure probe in the instructions for use for the DIAGNOdent pen.

Use:


Insert the Paro probe into the PA pocket parallel to the tooth surface.

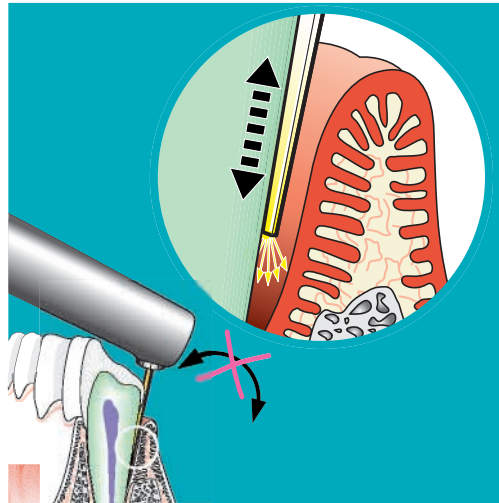

Display of the DIAGNOdent pen with the Paro probe:

Value	Meaning
>40	Concrements in the PA pocket.
5-40	Very small concretions Concrements next to the probe Potential root caries
<5	Clean PA pocket.

Calibration:

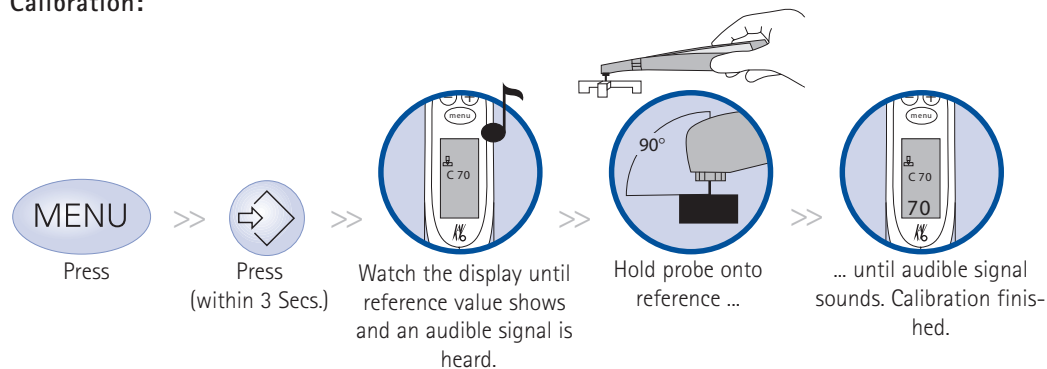

Those short operating instructions contain only the essential control functions. Before operating for the first time and in the case of operation by persons who are not familiar with handling this medical product, it is essential to read the Comprehensive Operating Instructions included in the delivery.



KaVo. Dental Excellence.

KaVo Dental GmbH

Bismarckring 39 • D-88400 Biberach

Tel. (49) 073 51/56-1500 • Fax (49) 073 51/56-14 88

Internet: www.kavo.com