SUNSTAR

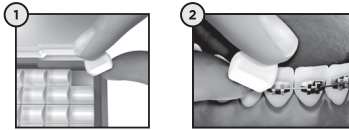
G.U.M.

ORTHO

 Sunstar Europe S.A.,
Route de Pallatex 11,
CH-1163 Etoy
ORTHOWAXLE321
Made in China
SunstarGUM.com

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EN Protection for wearers of orthodontic appliances

Product description: GUM ORTHO Wax is an orthodontic wax (medical device) composed of 5 pre-cut wax strips on wax paper in a plastic case available in 2 variants: unflavoured (Ref. 723) and mint flavour (Ref. 724).

Why use orthodontic wax?

Discomfort and irritations can be experienced when wearing orthodontic appliances and result from the repeated friction of braces onto oral mucosal tissues. Orthodontists recommend the use of orthodontic wax to form a protective barrier over brackets and wires to improve the comfort level during orthodontic treatment.

Indications: GUM ORTHO Wax has been specifically developed to protect oral mucosal tissues from lesions and relieve irritations caused by orthodontic appliances. It can be used as needed, both

in prevention and on irritating appliances. The period of use can extend up to 30 consecutive days. In case lesions or irritations persist, consult your orthodontist. GUM ORTHO Wax is for adults and children 7+ years old under adult supervision.

Contra indications: Do not use in the presence of hypersensitivity to any ingredients contained in the product.

Ingredients:

GUM ORTHO Wax: Microcrystalline Wax, Paraffin Wax, Paraffin Liquid Wax.

GUM ORTHO Wax Mint: Microcrystalline Wax, Paraffin Wax, Paraffin Liquid Wax, Aroma, Carvone, D-Limonene, Eugenol, Linolol.

Instructions for use: Wash and dry your hands. Dry the area of application for optimal adhesion. Break off a piece of GUM ORTHO Wax, warm it up and shape it between your fingers.

Apply the wax to cover wires and brackets by pressing gently. Remove the wax before eating or brushing your teeth. Insert a new piece of wax after. The wax can be kept in overnight. After using the wax, please discard it in the household waste.

Warnings and precautions: Keep out of the reach of children. GUM ORTHO Wax is intended for oral use only. Do not swallow. Storage conditions: +5°C to +40°C. Do not use the wax after the expiry date. Do not use if the device packaging integrity is damaged. If you notice any serious incident in relation with the use of the device, contact the manufacturer and the competent authority of your country. GUM ORTHO Wax has not been tested in pregnant or breast-feeding women. If you are taking medical products or are pregnant or breast-feeding, ask your doctor or pharmacist for advice before using GUM ORTHO Wax.

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