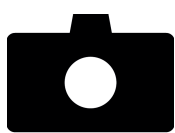
WARRANTY

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Privacy notice

EN

We process the personal data collected with this form (hereinafter „the data“) in order to ensure optimum support for our customers with regards to the device guarantee granted by us, and to be able to send them – as described in detail below – current information and offers on our products and services from time to time. Accordingly, METASYS Medizintechnik GmbH is responsible for compliance with the provisions of the data protection law.

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1. Purposes and legal bases of data processing

We shall use the given company master data (company, address and contact data, etc.) to register your company in our customer database. This shall in particular be used for the purpose of fulfilling contractual obligations in connection with the device guarantee granted by us. The legal basis for this is Art. 6 para. 1 item b) of the General Data Protection Regulation (GDPR). We shall process this data to be able to send our customers individual information and current offers on METASYS products and services from time to time. This data processing for the purpose of direct advertising shall also be in our legitimate interest. The legal bases for this are Art. 6 para. 1 items b) and f) GDPR.

2. Data security/passing on of data to third parties/duration of storage

The data you provide will of course be treated confidentially and protected by suitable technical and organisational precautions against loss and changes as well as unauthorised access by third parties. For internal administrative purposes, personal data of our customers are processed and stored in a central data management system of the METASYS Group, i.e. METASYS Medizintechnik GmbH and its EU affiliates. METASYS Medizintechnik GmbH is responsible for compliance with the provisions of this privacy policy and applicable data protection law. This data processing for administrative simplification and optimization is in the legitimate interest of METASYS Medizintechnik GmbH. The legal basis for this is Article 6 (1) lit. f) GDPR. We will not disclose your data to third parties outside the METASYS Group, unless we make use of specially selected and contractually obligated service providers (e.g. technicians, delivery services, printers) to fulfil the purposes stated in this privacy statement. The confidentiality and security of your data will be thereby be ensured in accordance with our privacy policy and applicable law. We shall store your data for as long as this is necessary for the aforementioned purposes and due to statutory documentation and retention obligations. If you no longer wish to receive information and offers from us, you can inform us at any time in accordance with section 3. In this case, we shall store your data for as long as this is legally required with regard to subsequent documentation or accountability obligations or with regard to the limitation of any mutual legal claims. The data will not be processed outside the European Union, unless the special data protection requirements of the GDPR are met.

3. Your rights

If and insofar as the respective legal requirements are met, you shall be entitled to request information from us about personal data or data processing concerning you (Art. 15 GDPR), rectification, erasure and restriction of personal data or data processing concerning you (Art. 16 to 18 GDPR) and transfer of personal data concerning you (Art. 20 GDPR). Furthermore, if the requirements of Art. 21 GDPR are met, you shall have a right of objection to data processing that relates to a „legitimate interest“ of the person responsible pursuant to Art. 6 para. 1 item f) GDPR or that is used for direct advertising. To exercise the aforementioned rights or if you wish to withdraw your consent to the processing of your personal data in whole or in part with effect for the future, you can contact us via the aforementioned contact details, in particular by using the e-mail address datenmanager@metasys.com. Art. 77 para. 1 GDPR also gives you the right of appeal to a supervisory authority, in particular in the Member State in which you are staying, in which you are working or where you suspect infringement of the GDPR to have taken place, if you believe that the processing of personal data concerning you is in contravention of the GDPR.

The data protection authority responsible for us in Austria is:

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Water Decontamination System
METASYS WEK
Equipment Logbook
Assembly, operation and maintenance

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Information on guarantee claims**EN**

METASYS delivers exclusively on the basis of the present terms and conditions, insofar as contractual partners have not struck a deviating agreement in writing.

1. Guarantee claims on our products are valid for a period from 12 to 36 months (depending on the product).
2. This guarantee includes all claims regarding material failures that affect the function of the device. Damages caused by wrong or improper handling or normal abrasion and the exchange of the collection amalgam container are excluded from the guarantee claim. Compulsory maintenance rates have to be met precisely. The producer reserves to himself the right to request the corresponding product documents to ascertain service rates.
3. To assert the validity of the guarantee claim it is necessary to return the attached installation proof to the producer right after the installation. Installation and return of installation proof must be completed within 24 months after date of sale from METASYS.
4. Both the installation and the required inspections have to be carried out by a technician trained by METASYS and have to be registered in the METASYS equipment logbook.
5. We point out that guarantee claims for amalgam separators can only be asserted if the device has been treated with the suction system disinfectant METASYS GREEN&CLEAN M2 For air/water separations and central suction systems guarantee claims can only be asserted if the devices have been treated with the suction system disinfectant GREEN&CLEAN M2 or the cleaner GREEN&CLEAN CL according to the directions of use. In case of water decontamination system WEK/WEK Light, the unexceptional use of the water decontamination preparation GREEN&CLEAN WK is essential for the validity of a guarantee claim. This regulation does also apply to the hygiene system H1, which must exclusively be operated with GREEN&CLEAN H1. Implied warranty remains unaffected of this.

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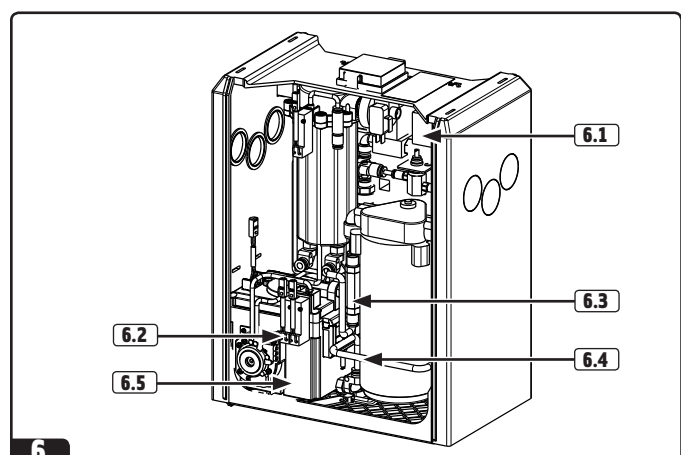
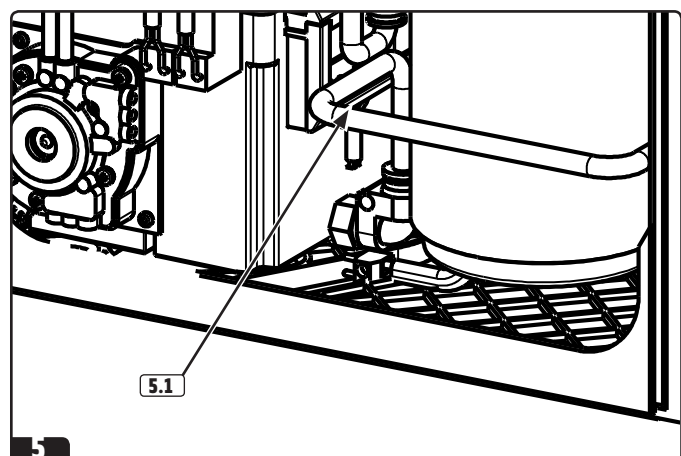
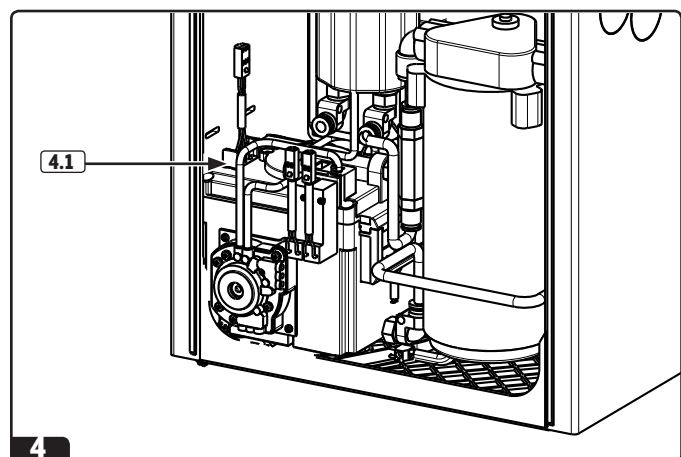
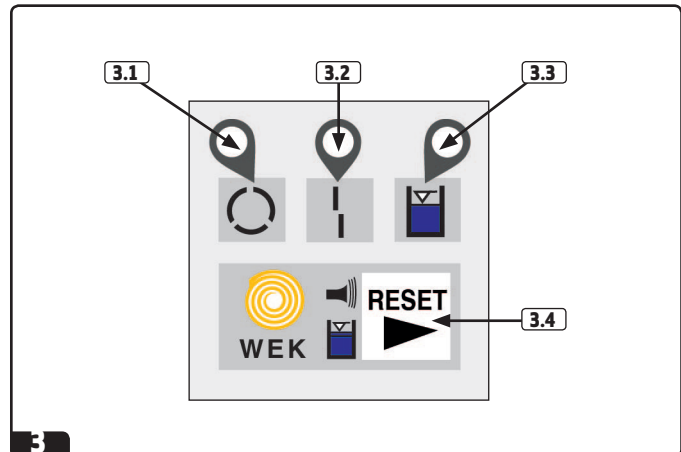
1. Index

The header on each page defines the user group particular information is aimed at.

Chapter	Page
1. Index	2
2. Explanation of the pictograms	2
3. General information	3
4. Application	4
5. Construction	4
6. Explanation of the type plate	5
7. Technical data	5
8. Functional description	6
9. Installation options	7
10. Hose connections	8
11. Electrical connections	8
12. Description of the control unit	9
13. Disinfectant	10
14. Changing the chemical bottle	10
15. Maintenance and cleaning	11
16. Service	11

2. Explanation of the pictograms

 Information

 Caution!

 General warning sign

 Refer to instruction manual/booklet

General information

Practice personnel, technicians

3. General information

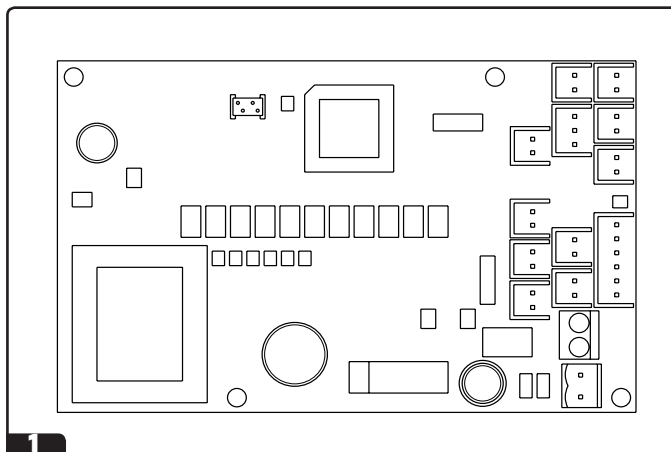
The safety, reliability and performance of the appliance is only guaranteed by METASYS if the following instructions are observed:

- Assembly, alterations or repairs may exclusively be carried out by authorized service personnel in compliance with EN Standard 60601-1 (International Standard for Medical Electrical Apparatus, in particular Part 1: General Rules for Safety).
- The electrical installation must comply with the regulations of the IEC (International Commission for Electrical Engineering).
- The apparatus must exclusively be used in conformity with the instructions for installation, operation and maintenance.
- Only original parts may be used for repairs or replacements.
- All the guidelines provided by the manufacturer on the correct use of the water decontamination system WEK must be followed.
- After commissioning, complete the proof of installation at the front page of this manual and send this to METASYS in order to define the warranty period.
- All inspection and service work must be entered into the device documentation at the first pages of this manual
- When requested by an authorized engineer, METASYS agrees to make all documents available for the use of technically qualified service personnel.
- METASYS accepts no responsibility for damages caused due to external factors, such as wrong installation, improper use of the apparatus or unauthorized technical intervention.
- Users must study equipment and assure themselves of its good condition before every use.


The equipment is not suitable for use in explosive or combustible environment.


EN


1

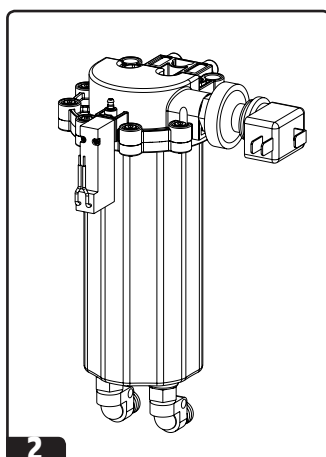2

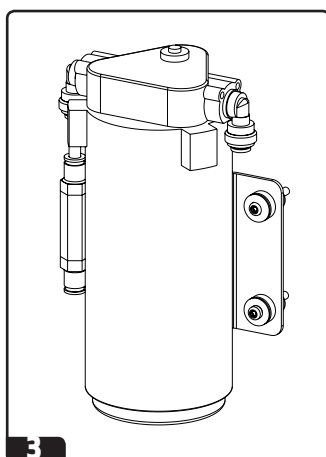
3

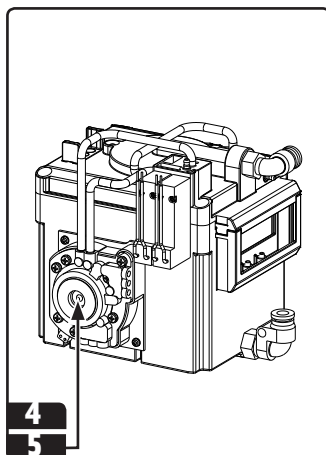
4

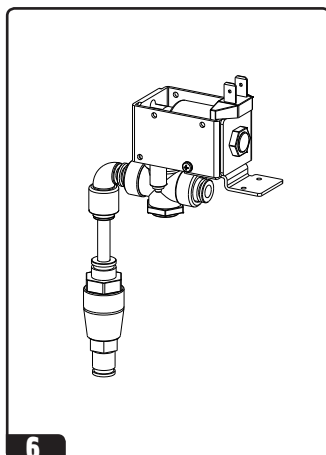
5

6

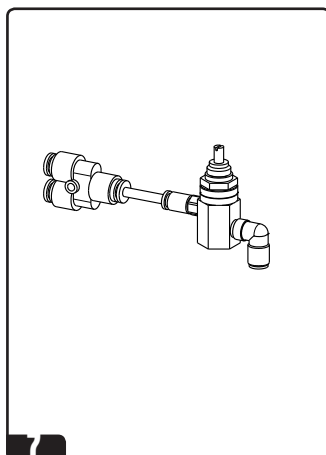7

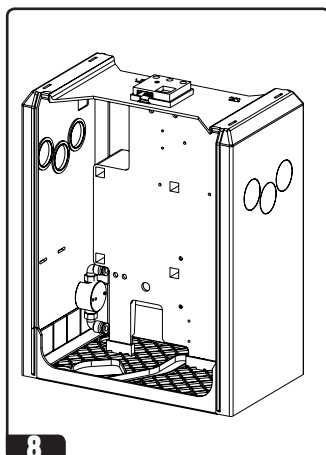8

4. Application

The METASYS water decontamination system (WEK) is a system which enables decontamination of the water supply and pipes using the special disinfectant GREEN&CLEAN WK.

In addition to its primary water decontamination function, the WEK also has a cascade (with DVGW-registration) to guarantee the separation of decontaminated water from tap water.

The WEK is used to supply water-bearing apparatuses and instruments (syringe, turbine, water glass filler, etc.) with decontaminated water.

The METASYS water decontamination system supplies all consumers of the dental unit. Their water consumption averages at:

- **syringe:** 100 - 130 ml / minute
- **turbine, handpiece and angle piece:** 50 - 70 ml / minute
- **water glass:** 150 ml / glassfull

5. Construction

To ensure optimal user and service friendliness, the water decontamination system consists of several **modules**:

- 1 Module 1** Electronic control system
- 2 Module 2** Pressure container; this generates the water pressure. In addition to the container, this module contains a magnetic valve, a pressure switch and a check valve
- 3 Module 3** Membrane pump; includes pump and check valve
- 4 Module 4** Mixing container; here, water is mixed with the disinfectant (after the cascade section). This module includes the container, filling and overflow sensor, 2 magnetic valves and the disinfectant sensor
- 5 Module 5** Double membrane pump and membrane break sensor
- 6 Module 6** Water unit; consists of a flow rate limiter and dosing valves
- 7 Module 7** Compressed air unit; this is a pressure regulator
- 8 Module 8** Housing (optional); consisting of housing, mounting plate, main water tap with prefilter (80 µm), transformer and main tap

Explanation of the type plate · Technical data

Practice personnel, technicians

6. Explanation of the type plate

9 See illustration


The type plate is attached to the top of the auxiliary housing.

- 9.1 Equipment type name
- 9.2 Mains supply data
Built-in unit (UN = 24 V AC)
Floor model (UN = 230 V AC)
- 9.3 Maximum power consumption
- 9.4 Maximum water flow rate
- 9.5 Permitted water pressure
- 9.6 Serial number
- 9.7 Address of manufacturer
- 9.8 CE conformity mark

7. Technical data

Versions	floor model / built-in unit
Voltage	230 V AC (floor model)/ 24 V AC (built-in unit)
Frequency	50/60 Hz
Maximum power consumption	100 mA (floor model)/ 1.2 A (built-in unit)
Permitted water pressure	1 - 6 bar
Permitted air pressure	3.5 - 8 bar
Operating pressure (water)	2.5 bar
Operating pressure (air)	3.5 bar
Maximum water flow rate	1 l/min
Disinfectant	GREEN&CLEAN WK hydrogen peroxide, 2% solution
Mixing ratio	1:85 standard setting, 1:42 intensive decontamination
Working solution	235 ppm
10 see figure Dimensions (H x W x D)	335 x 265 x 160 mm

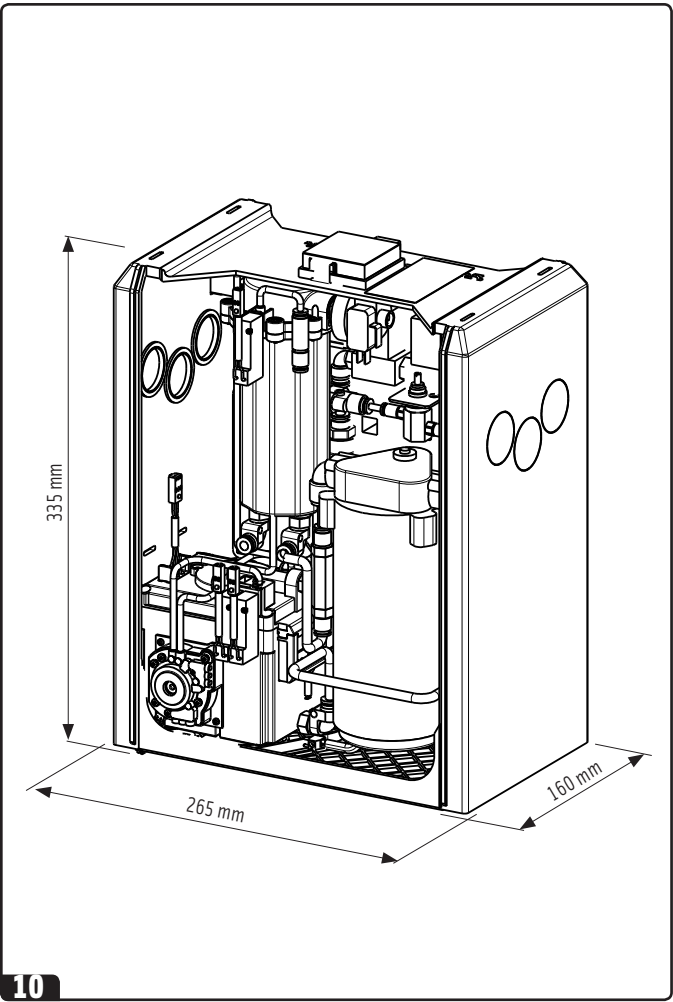
EN

Functional description

Practice personnel, technicians

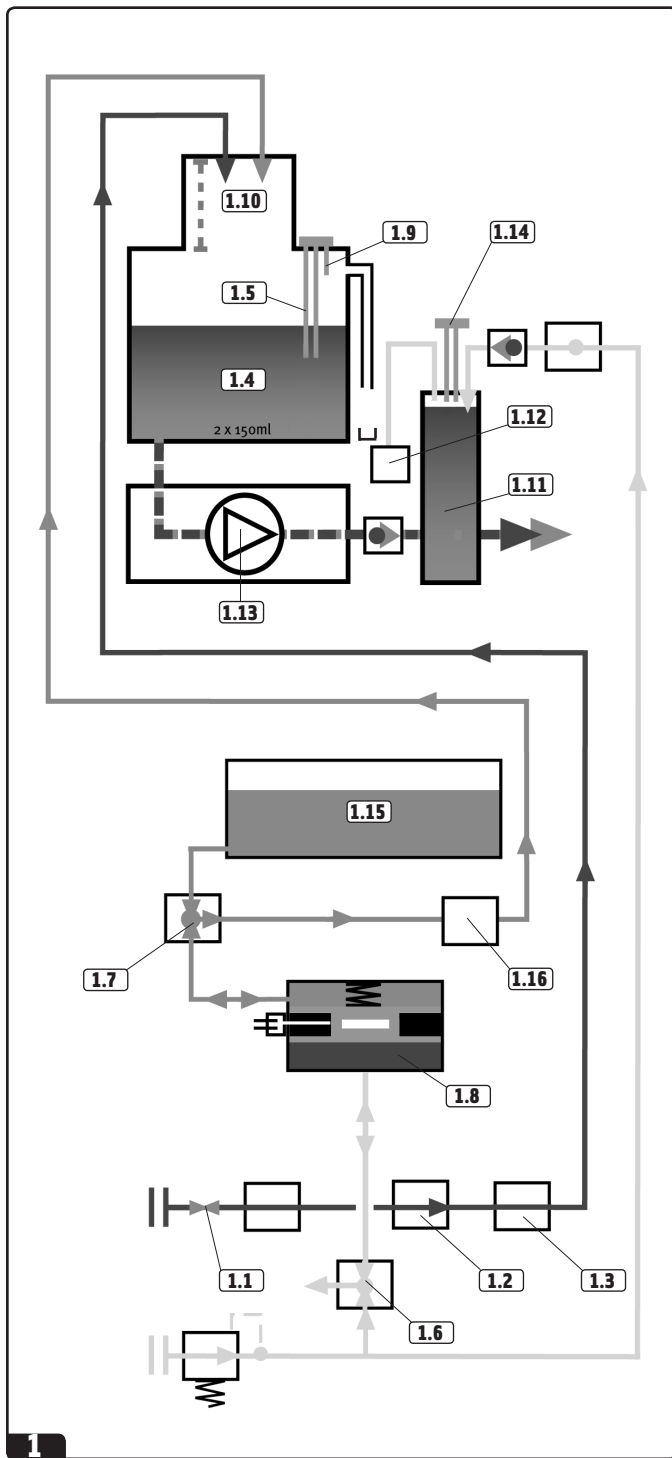EN

8. Functional description

1 See illustration

When the water tap **1.1** is opened and the system is switched on, the dosing valve **1.2** and the safety valve **5 1.3** open and 150 ml water are supplied to the mixing container **1.4**.

The water level in the mixing container is monitored by an electronic probe A **1.5**. Once the contact on the probe is set off, the magnetic valves **2 1.6** (compressed air) and **3 1.7** (GREEN&CLEAN WK) open. This activates the compressed air driven double-membrane pump **1.8**, causing approx. 1.75 ml hydrogen peroxide to be withdrawn from the chemical bottle and added to the mixing container.

If the water level drops below probe A, the dosing valve opens again until overflow probe B **1.9** is reached. By the time the water reaches this probe, the mixing container has been filled with 300 ml of water. Magnetic valves **2** and **3** open again, and add another 1.75 ml of chemical to the mixing container. This ensures that the required concentration of disinfectant in the solution is continuously maintained.

In the mixing container, the water inlet is in the form of a cascade **1.10**, ensuring that the WEK is separated from the water supply as required by the DVGW (German Gas and Water Association) (DIN 1988, Part 4). This ensures that water contaminated with microorganisms or mixed with chemicals cannot flow back into the water supply system. If a water-fed consumer is activated, the decontaminated water flows from the mixing container, via the pressure container **1.11** to the appropriate instrument.

Naturally, this causes the water level in the pressure container to drop, together with the water pressure. A pressure switch **1.12**, fitted to the pressure container and adjusted to 2.5 bar, activates the membrane pump **1.13** which is responsible for emptying the mixing container, filling the pressure container and maintaining the required water flow pressure.

An air cushion in the pressure container compensates the water level. If necessary, the air cushion is topped up with compressed air (magnetic valve **4**) - all this is done to ensure a stable water jet for the instrument. The air cushion probe C **1.14** ensures that the air pressure is kept constant.

The disinfectant is withdrawn from the chemical bottle **1.15**.

A disinfectant sensor **1.16** controls the availability of the chemical. If no more chemical can be withdrawn (e.g. the bottle is empty), an error message occurs.

Please note that only METASYS GREEN&CLEAN WK may be used, as the electrical sensors are adjusted to this product!

Installation options

Technicians

9. Installation options

The modular design of the WEK Water Decontamination System allows the following installation options:

2 Integration into the treatment unit

Direct integration of the WEK Water Decontamination System into the treatment unit is the preferred option in order to keep the water and compressed air pipes supplying the WEK as short as possible.

Care should be taken to ensure vibration-free attachment of the unit to weight-bearing parts of the treatment unit.

Filter:

If not already integrated into the treatment unit, a filter with maximum mesh size of 80 µm must be applied before the water supply connection of the WEK.

3 Integration in the auxiliary housing

If the direct integration of the water decontamination system into the dental treatment unit is not possible, it can be installed as a floor model with a shapely housing.

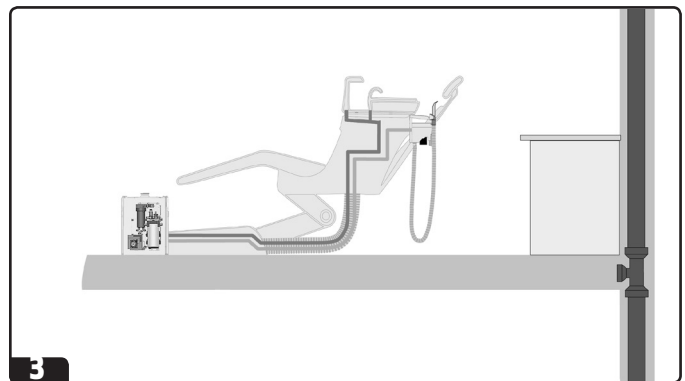
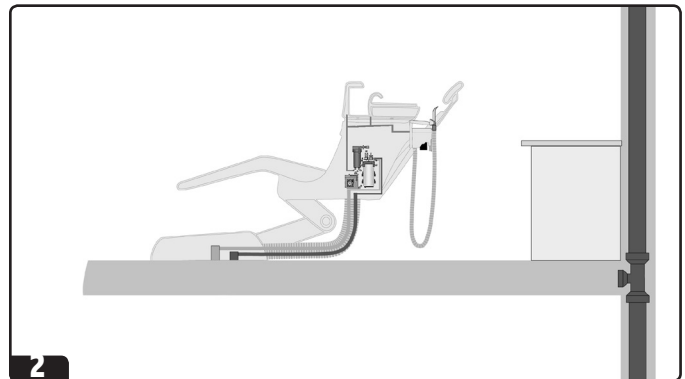
The housing requires only a minimum amount of floor space and should be mounted onto the junction box of the treatment unit. The external display and a transformer are already incorporated into the housing.

Main switch:


It must be ensured that the power supply is connected downstream of the main instrument/ practice switch.

Please note that the transformer is still energised if the main switch of the unit is turned off! The unit has to be disconnected from the power supply with the main instrument/practice switch, before any installations or repairs are performed!

EN

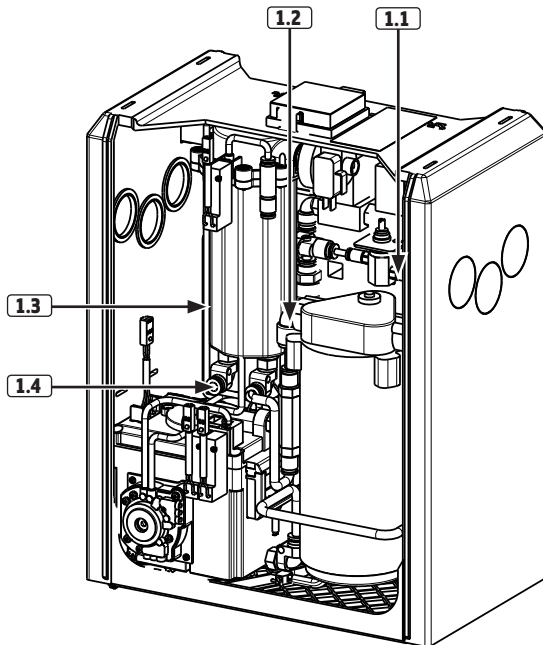10. Hose connections

1 See illustration

1.1 Compressed air connection

Permitted pressure at entry: 3.5 - 8 bar

PUR tubing

I.D. = 2.5 mm / O.D. = 4 mm

1.2 Water connection (fresh water)

Permitted pressure at entry: 1-6 bar

PA tubing (with KTW authorisation)

I.D. = 4 mm / O.D. = 6 mm

1.3 Chemical bottle connection

PVC tubing

I.D. = 2 mm / O.D. = 4 mm

1.4 Connection to the water outlet (supply of instruments)

PUR tubing

I.D. = 4 mm / O.D. = 6 mm

11. Electrical connections

2 See illustration

2.1 Overflow sensor (yellow)

2.2 Pressure switch (red)

2.3 Mixing container probe (yellow/red/black)

2.4 Disinfectant sensor (green)

2.5 Level monitor in the pressure container (blue)

2.6 Air cushion valve on the pressure container (black) MV 4

2.7 External display

2.8 Chemical valve (black) MV3

2.9 Water safety valve 5 (brown)

2.10 Pump motor connection

2.11 Membrane break sensor (yellow)

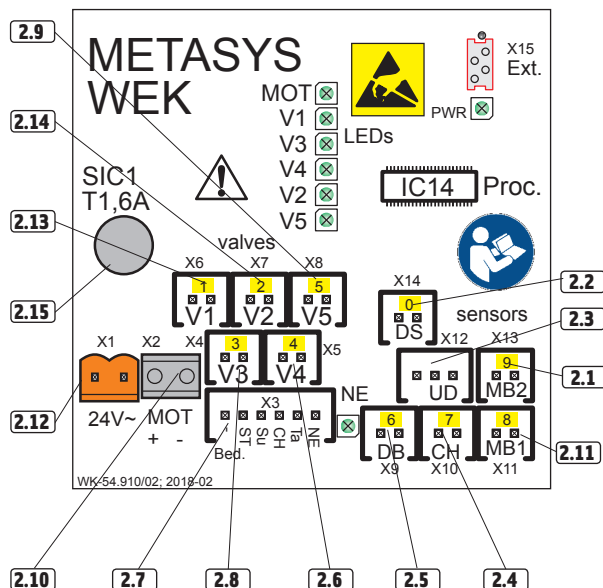
2.12 Power supply plug 24 V AC (orange)

Warning: The power voltage must be supplied by a transformer which meet the specifications of EN 60601-1 and EN 61558-2-6.

2.13 Water dosage valve 1 (brown)

2.14 Compressed air valve (black) MV2

2.15 Main fuse MST250/T 1,6 A/UN 250 V/ ICN 35 A (replace with same type only!)

Description of the control unit

Practice personnel, technicians

12. Description of the control unit

3 See illustration

- 3.1** Control light 1: Ready for use (green)
- 3.2** Control light 2: Error (red)
- 3.3** Control light 3: Disinfectant sensor (yellow)
- 3.4** Alarm RESET button

4 Switch on/checking routine: The green control signal **3.1** flashes

After having switched on the WEK it automatically performs a test routine. During this procedure control signal 1 **3.1** is blinking fast. If an error occurs, the unit is not activated and the error is shown at the operating unit (see below). After the successful completion of the test routine, control signal 1 lights constantly and the unit is ready for operation.

5 The green control signal **3.1** and the yellow signal **3.3** are illuminated, as well as sound of the alarm:

Cause: Disinfectant sensor indicates that the process water is not being decontaminated!

Action: Change the chemical bottle (see page 10)!

If the chemical bottle cannot be changed immediately, the RESET button must be pressed to deactivate the continuous alarm. The unit remains fully functional and does not interrupt the dentist's work. However, a short signal tone still sounds periodically. In case the error had been caused by an air bubble it will be gone at the next attempt.

However, no decontamination takes place until the empty chemical bottle is replaced with a full one (see page 10).

6 The green control signal **3.1** is illuminated and the red signal **3.2** flashes, the alarm tone sounds:

Cause: Level monitor in the mixing container is contaminated, or no water supply!

Action: First of all, check if the main water tap and water inlet pipe are open! Clean or replace the mixing container probe (see page 11).

Consequence: All valves and the pump are switched off. The alarm tone can be switched off by pressing the RESET button. After resolving the problem the WEK can only be reactivated by switching the unit off and on.

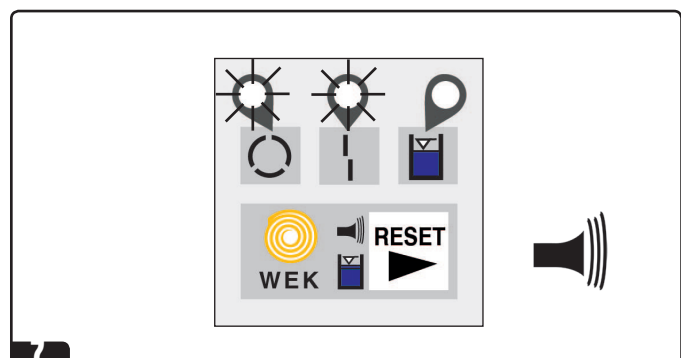
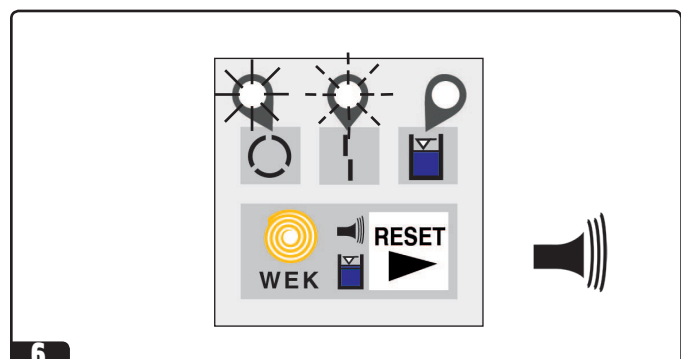
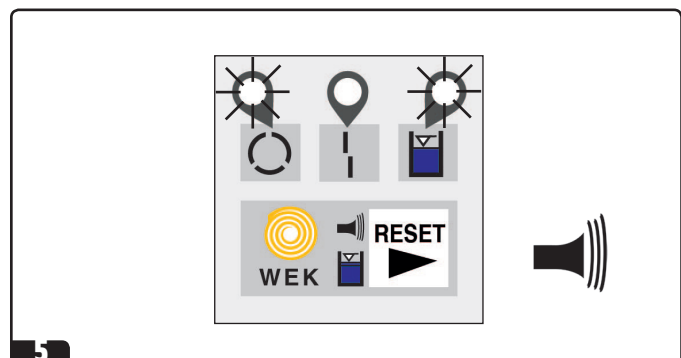
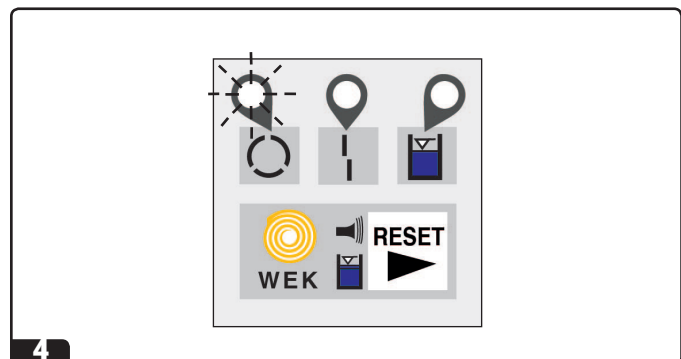
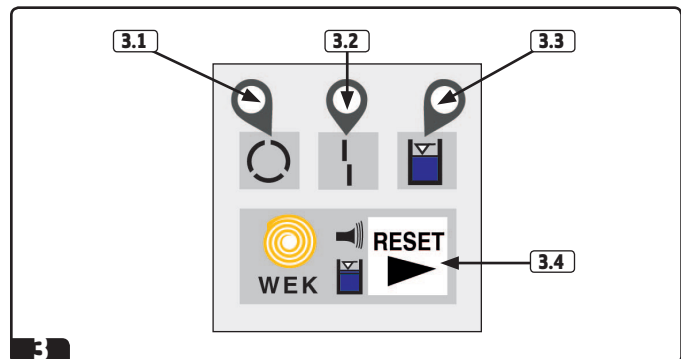
7 The green control signal **3.1** and the red signal **3.2** are illuminated, the alarm tone sounds:

Cause: Defect of the double membrane pump, or overflow!

Action: Clean the overflow sensor. Check the double membrane pump and the electronic control system (see page 11).

Consequence: All valves and the pump are switched off. The alarm tone can be switched off by pressing the RESET button. After resolving the problem the WEK can only be reactivated by switching the unit off and on.

EN

Disinfectant · Changing the chemical bottle

Practice personnel, technicians

EN

1.1

1.2

13. Disinfectant

 See illustration

GREEN&CLEAN WK, a 2% hydrogen peroxide solution with added silver, is used as the disinfectant. Due to their potent bactericidal and fungicidal action, hydrogen peroxide-based concentrates are particularly suitable for reducing microorganism levels in process water. The ratio of disinfection efficacy and potential risks of hydrogen peroxide are excellent, which means they present no risk to the patient.

During each disinfecting cycle of the METASYS water decontamination system, a designated amount of disinfectant is added to the 150 ml of water in the mixing container. This is how the 0.0235% concentration required for broad-spectrum disinfection is prepared.

Once the GREEN&CLEAN WK bottle is completely empty, an error signal appears on the external display, stating that no more disinfectant can be drawn. The empty bottle needs to be disconnected from the cannula of the water decontamination device and replaced with a full one.

Please note that only GREEN&CLEAN WK provides the concentration required for broad spectrum disinfection.

GREEN&CLEAN WK - refill sets		Order numbers
1.1	4 x 750 ml bottles	60040100-1
1.2	6 x 1000 ml bottles	60040101

The GREEN&CLEAN WK refill set can be ordered directly from your dental depot!

Safety precautions: Keep out of the reach of children. In case of contact with eyes, rinse immediately with plenty of water. In case of contact with skin, wash immediately with plenty of water. Store GREEN&CLEAN WK at room temperature (or 5°C - max. 30°C) and protected from light!

Composition: water, hydrogen peroxide, silver and less than 0.1% stabilisers

Please refer to the safety data sheet for further information!

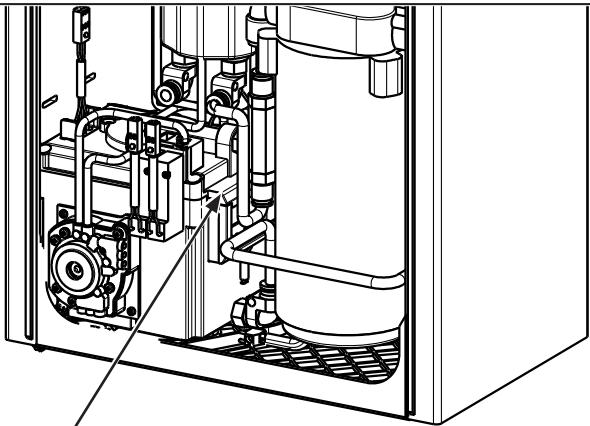
14. Changing the chemical bottle

 See illustration

Remove the empty bottle from the device. Pull gently on the closure and the attached tube reaching into the bottle to remove the bottle **2.1**.

After having removed the screw cap of the new bottle insert the tube into the bottle and press the closure onto it **2.1**.

Afterwards place bottle back into the device. Ensure that tube is connected to the chemical port **2.2**.

2.2

2.1

15. Maintenance and cleaning

3 Additional programs

Automatic emptying: To empty the unit during operation keep the RESET button **3.4** pressed for approx. 8 sec. (control signal 1 **3.1** flashes) until a second acoustic signal is activated. Then the green control signal 1 flashes slowly. Reservoir and pressure container are empty as soon as no more water comes out at instruments or water glass filler. The WEK can now either be switched off, or the normal operation can be activated by keeping the RESET button pressed for another 8 sec.

Intensive decontamination: If required, the concentration of the chemical may be increased. The intensive decontamination can be activated by holding down the RESET button **3.4** (control signal 1 **3.1** flashes) for approx. 4 sec. until the first acoustic signal is activated. While this procedure is taking place control signal 1 **3.1** flashes and signal 3 **3.3** is illuminated. With the next chemical dosing operation, 3 doses of concentrate will be delivered, followed by 2 doses each in the next 4 dosing operations. No chemical will be added to the final dosing to ensure normal concentration after the intensive decontamination. Control signal 3 stops after the intensive decontamination and a short acoustic signal is activated. The WEK returns into normal operation.

4 Remove water level probe from the mixing container and clean or replace (see illustration)

- Switch off main switch.
- Remove bottle.
- Remove retaining plate by lifting.
- Carefully pull probe up and out.
- Clean or replace probe. Check the electronic control system.
- Afterwards place the probe back into the mixing container **4.1**
- Snap in retaining plate.
- Place chemical bottle into unit.
- Switch on main switch.

5 Check overflow sensor at the mixing container (see illustration)

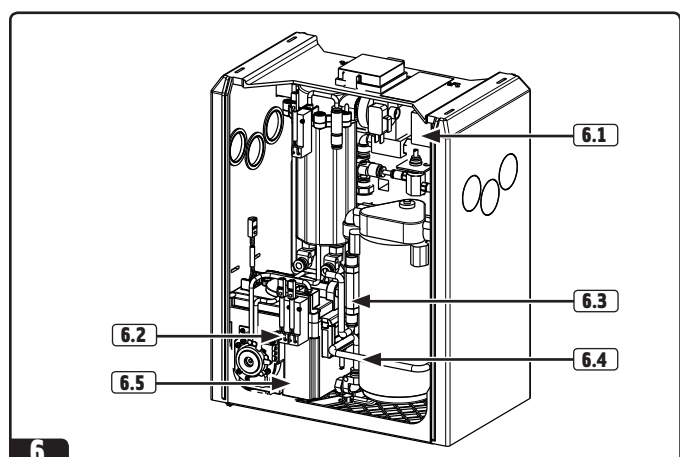
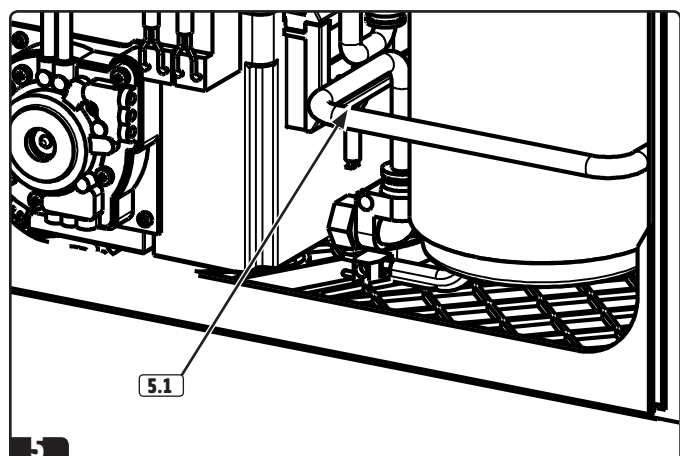
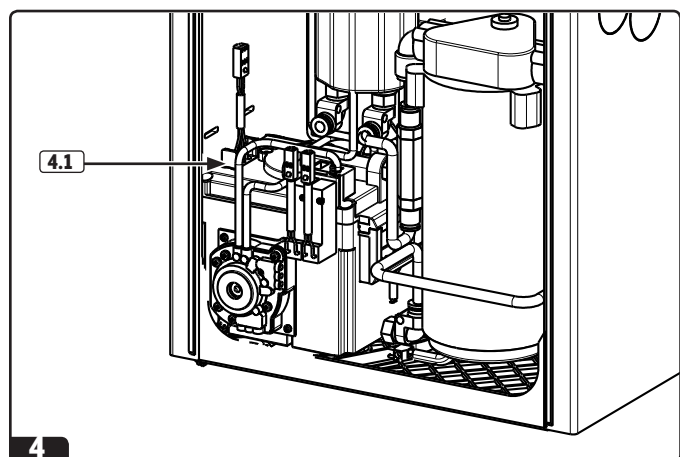
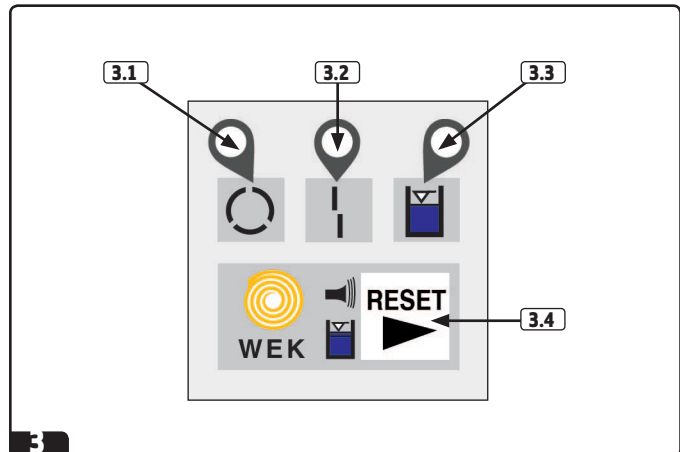
- Switch off main switch.
- Clean/dry overflow sensor **5.1**. Check electrical connectors of the electronic control system (see page 8)
- Check double membrane pump and the electronic control system.
- Switch on main switch.

16. Service

6 Annual service / 4-year service

An annual service of the WEK water decontamination system (service kit order no. 50050115) is required. This service includes a complete function check of the device, inspection (replacement, as necessary) of the water dosing valve **6.1**, replacement of the magnetic valves **6.2**, the water non-return valve **6.3**, the tubes **6.4** and the fine filter. Additionally, an inspection and cleaning of the mixing container **6.5** and its probes (see points 4 and 5) must be carried out.

Every 4 years, the 4-year service (service kit order no. 50050129) must be carried out wherein, additionally, the lid of the mixing container must be replaced.

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