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Product Description

Mucopren® Soft is a permanently soft durable relining material for removable dentures based on vinyl polysiloxane.

Indications / Techniques

Mucopren® Soft can be applied directly or indirectly

- for permanently soft linings
- for compensating for areas of mucous membrane with varying degrees of resiliency
- for relieving sharp bony ridges
- for relieving the operation site after placing implants
- for relining temporary restorations
- for conditioning the mucous membranes in cases with denture sores
- for fabricating and relining obturators

Warnings

Do not use Mucopren® Soft relining material as an impression material.

Do not use Mucopren® Soft products with other reliners, sealants or adhesives.

Cautions

Do not use after expiration date.

Handling instructions for relined dentures are in this sheet.

Do not leave remains of soft liner in the mouth.

Do not swallow this material! If swallowed: consult a medical doctor if problems arise or persist.

Avoid contact with eyes. If accidental contact with the eyes occurs, rinse immediately and thoroughly with an eye wash or water. Consult a medical doctor if problems arise or persist.

The product does not normally cause allergic reactions. However, for sensitive persons, an allergic reaction to the product cannot be ruled out.

Also, if used on patients whose immune system is impaired, or on those who are suffering from a variety of other diseases (such as diabetes mellitus) or those who are taking medications including antibiotics or cortisone, it has been observed in rare cases that micro organisms (e.g. candida albicans) may colonize the soft lining. In such cases, whenever the lining of the dentures is replaced, soft lining should not be used since the recolonization by micro organisms is likely if the patient status remains unchanged.

Increased colonization by microorganisms and the development of denture stomatitis have been observed in patients due to inadequate oral hygiene and improper cleaning of soft-lined dentures.

Please instruct the patient about adequate oral hygiene and provide him with the care instructions from the base set.

Solutions used prior to the application of Mucopren® Soft (e.g., retraction solutions, mouth rinsing solutions) may interfere with the setting reaction of the relining material and must be removed by thorough spraying and drying.

If gloves are worn when processing the material, they should first be tested for compatibility using a sample. Some types of gloves can disturb the activity of the catalyst.

Should the silicone soft liner require adjustment, relieve the area liberally and apply another coat of Mucopren® Soft. Brush two coats of adhesive onto the exposed areas of acrylic only.

When handling chemicals the usual precautions must be taken.

Do not allow the material to enter the sewer or water system to avoid environmental contamination.

Avoid contact with clothing, since the material cannot be removed by dry cleaning.

Note

To ensure optimal relinings, the temperature of the material should not deviate from 23°C/74°F before applying. Otherwise, working and setting times will be affected. The water temperature and time in the water are critical factors in ensuring an optimum bond between Mucopren® Soft and PMMA.

Please Note – Mucopren® Adhesive

Close the bottle of Mucopren® Adhesive immediately after use.

Should the contents of the bottle be spilled, it is advisable to ventilate the area and absorb the spilled adhesive with universal binding agent.

The adhesive contains ethyl acetate.

Instructions on the Use of Cartridge Material in the Applyfix 4 Dispensing Gun DS-50 1:1 / 2:1

Insert the cartridge into the Applyfix 4 dispensing gun DS-50 1:1 / 2:1. Ensure that the notches on the cartridge base are pointing down. The clasp will not close if the cartridge is not properly inserted into the dispensing gun.

Remove the cartridge cap. The cap can be replaced after initial use.

Insert plunger into cartridge and dispense a small amount of reline material until equal amounts are dispensed at the same rate.

Install a mixing tip on the cartridge, and turn cap 90 degrees in a clockwise direction to lock in place.

Apply Mucopren® Soft uniformly with a dispensing gun and insert it into the patient's mouth immediately.

Replace the plug after use. Before using the cartridge again, remove the cartridge cap or the filled mixing tip and check the cartridge outlet openings to ensure that no polymerized materials have sealed the openings. Clear any clogged openings and dispense a small amount of reline material until equal amounts are extruded at the same rate.

Install a new mixing tip and continue as described above.

Cleaning and Disinfection of the Applyfix 4

Remove the cartridge.

Remove any uncured paste using a cloth soaked in alcohol.

The Applyfix 4 dispensing gun can be used several times if properly disinfected. Kettenbach recommends exchanging the gun if there are visible traces of use. Disinfection can be carried out using a commercially available dipping disinfectant. Kettenbach recommends the use of glutaraldehyde-based solutions.

Instructions for Use for the Direct and Indirect Methods

Mucopren® Soft was developed specifically for the use of cold and hot polymerizing denture base plastics. Wash the denture base thoroughly with water and dry it. Use a bur to reduce the surface to allow for the thickness of soft liner required. To create a neat vestibular juncture between the soft liner and denture acrylic, it is advisable to trim a right-angled periphery as shown in Figure 1, page 90.

Once the surface has been prepared and roughened, brush on a thin coat of Mucopren® Adhesive, wait 30 seconds before applying a second coat, and allow it to dry in air for at least 90 seconds. Ensure that the adhesive is applied **beyond the prepared periphery**.

Apply Mucopren® Soft uniformly with a dispensing gun (to prevent air voids being entrapped, keep the mixing tip in the material) and insert it into the patient's mouth immediately. The active and passive functional excursions can then be carried out. Processing: Once the denture has been removed from the patient's mouth, immerse the denture and soft lining in water at $\geq 50^{\circ}\text{C}/122^{\circ}\text{F}$ for at least 15 minutes.

Dentures which are not made of standard self-curing acrylic should be immersed in lukewarm water ($30^{\circ}\text{C}/86^{\circ}\text{F}$ – $35^{\circ}\text{C}/95^{\circ}\text{F}$) for 2 hours.

Once the material has cured, cut the excess material off with scissors, a scalpel or rotary instrument to create a neat juncture between the denture acrylic and softliner (refer to Figure 2, page 90). Cross-cut steel burs for soft linings (e.g. Figur 82060 from Busch) are useful for bulk reduction. Final finishing can be carried out with the polishing discs (supplied with the basic set). Trimmed areas of acrylic can be polished as usual. Do not use polishing paste that leaves residual grease. Always clean the surfaces with hot water (steam cleaner) and dry them before sealing.

Lastly, apply a thin layer of Mucopren® silicone sealant to the prepared area of soft liner. It is advisable to place a small amount of Mucopren® silicone sealant into a suitable vessel (e.g. the plastic one supplied in the pack) and then apply it with a brush within 90 seconds (see Figure 3, page 91). The silicone loses its adhesive properties after 6 minutes and 30 seconds. Please note: The silicone sealant does not attain its final mechanical properties until it has been at mouth temperature for several hours.

When relining indirectly, apply adequate amounts of separating agent to the model (use a separator which forms a microfilm e.g. Dentsply Isolant®). As agar-agar based separating agents may form a culture medium for micro organisms such as candida albicans, they must not be used. The relining jig or flask can be placed in a pressure vessel with water at a tem-

perature of $\geq 50^{\circ}\text{C}/122^{\circ}\text{F}$ for at least 30 minutes to ensure a uniform setting time.

Storage Conditions

Store in a dry place, protected from sunlight and at room temperature ($10^{\circ}\text{C}/50^{\circ}\text{F} - 25^{\circ}\text{C}/77^{\circ}\text{F}$); avoid extreme temperatures. Do not use after the specified expiration date.

Disinfection

Soft lined dentures can be disinfected (immersion disinfection).

The cross-cut steel bur for permanent soft relining materials (Figur 82060 from Busch) that is included in the basic set is manufactured from normal steel and is therefore not suitable for reprocessing in a steam steriliser.

It is also important for normal steel to ensure that disinfectants and cleaning agents are used with corrosion protection.

Disposal

The empty cartridges/mixing tips/applicator tips and set material can be disposed in the household rubbish.

Trademarks

Mucopren® is a registered trademark of Kettenbach GmbH & Co. KG.

Note

Selected Kettenbach materials/products are available in certain markets only.

For dental professional use only. Caution: Federal (U.S.A.) Law restricts this device to sale by or on the order of a dentist.

If serious incidents occur that are related to the product, please immediately contact the manufacturer and the competent authority.

Limitation of Liability

Except where prohibited by law, Kettenbach GmbH & Co. KG will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence, or strict liability.

The information provided for Kettenbach products is based on comprehensive research and experience in application technology. Results are furnished to the best of our knowledge, subject to technical changes within the framework of product development. However, users must comply with and consider all recommendations and information in connection with any use.

Updated: November 24, 2021

Product Overview

Product name	ISO 10139-2	Mixing ratio and total content volume per unit	Mixing technique	Mixing element
Mucopren® Soft	Type A	1:1, 50 ml cartridge	Applyfix 4 dispensing gun DS-50 1:1 / 2:1	mixing tip, green, Ø 6.5 mm
Mucopren® silicone sealant	—	1:1, 50 ml cartridge	Applyfix 4 dispensing gun DS-50 1:1 / 2:1	mixing tip, blue, Ø 3.2 mm

Technical Data

Product name	Mixing ratio	Working time at 23°C / 74°F ≤	Working time at 35°C / 95°F ≤	Intraoral setting time at 35°C / 95°F ≥	Total setting time ≥	Hardness (approx.) Shore	Linear dimensional change (approx.) %	Elastic-Recovery test (approx.) %	Strain in compression (approx.) %
Mucopren® Soft Direct reline	1:1	2 minutes 15 seconds	not applicable	3 minutes 15 seconds	5 minutes 30 seconds*	A 28	– 0.20	99.5	9.4
Mucopren® Soft Indirect reline	1:1	2 minutes 15 seconds	not applicable	not applicable	> 30 minutes in pressure vessel	A 28	– 0.20	99.5	9.4
Mucopren® silicone sealant	1:1	1 minute 30 seconds	not applicable	not applicable	6 minutes 30 seconds at 23°C/74°F	A 33	– 0.20	—	—

* Total setting time (removal time from mouth) from start of mix

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DEAR PATIENT,

English

The bottom of your denture has been fitted with a permanently soft lining to make it as comfortable as possible for you to wear. The soft lining does not smell or taste at all.

Please do not eat for the first two hours or clean your denture by hand, e.g. with a toothbrush, for the first day. In individual cases, the consumption of coffee, carrot juice and red wine might cause discolorations of the denture.

You can help make the soft lining last longer by caring for it carefully. Rinse your denture daily in running water before cleaning it with any established brand of denture cleanser based on active oxygen (e.g. Blend-a-dent 2-Phase, Corega Tabs or Kukident). This reduces colonization by microorganisms. Then carefully clean your denture with a soft toothbrush and your usual toothpaste. Substances that do contain alcohol are not suitable for cleaning your dentures as they can have an adverse effect on the permanently soft lining.

Once you have cleaned your denture, rinse your mouth thoroughly and replace the denture in your mouth. Alternatively, keep your denture in water or a dry place – depending on what your dentist advises. In any case, follow your dentist's advice.

Should parts of the soft lining become detached due to overloading, your dentist can replace them.

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