Gebrauchsanweisung | Instructions for use |
Mode d'emploi | Modo de empleo | Modalità d'uso

Orthalgenat® Duo
MINT

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Dear customer,

Thank you for choosing a quality product from Dentaureum.

It is essential to read these instructions carefully and adhere to them to ensure safe, efficient use and ensure that you and your patients gain full benefit.

Instructions for use cannot describe every eventuality and possible application. In case of questions or ideas, please contact your local representative.

As our products are regularly upgraded, we recommend that you always carefully read the current Instructions for use supplied with the product and stored in the internet at www.dentaureum.com, even though you may frequently use the same product.

1. Manufacturer

Dentaureum GmbH & Co. KG | Turnstr. 31 | 75228 Ispringen | Germany

2. General information

Orthalgenat® Duo is a fast-setting impression material with significantly reduced dust generation. It is also suitable for taking impressions on multi-bracket patients due to its particularly high elasticity.

Dust-free. Lead-free. Cadmium-free. 1 kg is sufficient for 85 average-sized impressions.

Safety information: Orthalgenat® Duo contains diatomite.

If used incorrectly, dust may be generated.

Avoid inhaling dust.

Observe the safety data sheet.

3. Application

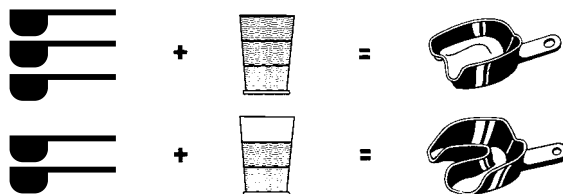
Dosage:

To achieve a consistent mixing ratio, turn the container over 3–4 times daily before removing the powder, or stir the material with the spatula.

The measuring beakers provided guarantee the best possible dosage and simplify working with the powder:

Fill the scoop with Orthalgenat® Duo powder. Then use a dry spatula to level the scoop off and remove surplus powder. Do not compress the powder!

1 scoop (6 g) Orthalgenat® Duo equals one graduation mark (12.6 ml) on the water measuring beaker.

Mix the powder and water in a clean mixing bowl.

Manual mixing time	Mix powder and water to a homogeneous mass for 30–45 seconds with the mixing spatula.
Machine mixing time	Close the mixing bowl and shake briefly but vigorously. Then machine-mix for about 10–14 seconds. Follow the manufacturer's instructions.
Mixer	We recommend the Mikrona Migma 200, 12 seconds mixing time. Follow the manufacturer's instructions.
Processing time	Approx. 1 minute at 23 °C/73 °F water temperature and 23 °C/73 °F room temperature
Initial setting time	Machine mixed: approx. 1.5–2 minutes Manually mixed: approx. 2–2.5 minutes The data refer to the use of deionized water at a temperature of 23 °C/73 °F. Lower temperatures lengthen the processing and setting time, while higher temperatures and increased water hardness shorten it. For a constant water temperature we recommend storing the water in the dosing bottle REF 150-000-00.
Minimum time in mouth	Approx. 40 seconds
Disinfection	Put the hardened impression in a standard disinfection solution; please observe the manufacturer's instructions. We recommend MD 520 (Dürr Dental, Germany).
Model fabrication	Rinse the hardened impression under running water (disinfect beforehand) and fill with plaster (e.g. Rapidur® type 4 or Moldano®, type 3). If the impression cannot be processed immediately, it must be stored damp. We recommend wrapping it in a cloth that has been soaked in water and storing it in a plastic bag. We recommend the impression is cast within 2 days.
Storage	Orthalgenat® Duo should be stored dry and at room temperature (max. 25 °C/77 °F). It has a shelf life of 3 years from the date of manufacture. Always close the container after each use. If the container is left open over a long period of time, the powder will get damp which will lead to a shorter setting time.

4. Additional information

Orthalgenat® Duo is for single use only.

Should the user and/or the patient become aware of serious problems arising from the use of the product, it is important that the manufacturer and the competent authority in the country in which the user and/or the patient is resident is informed accordingly.

5. Delivery program

Orthalgenat® Duo mint

REF 150-501-20	1 kg	Container
REF 150-509-20	9 x 1 kg	Cardboard box
REF 150-520-20	20 x 1 kg	Cardboard box

Dosing bottle for liquids

REF 150-000-00	500 ml	Dosing bottle
----------------	--------	---------------

Orthalgenat® Ex

For removing residual alginate and cleaning the impression trays.

REF 150-010-00	1.5 kg	Container
----------------	--------	-----------

6. Quality information

Dentaurum ensures faultless quality of its products. These recommendations are based upon Dentaurum's own experiences. The user is solely responsible for the processing of the products. Responsibility for failures cannot be taken, as we, Dentaurum, have no influence on the processing on site.

7. Explanation of symbols used on the label

Please refer to the label. Additional information can be found at www.dentaurum.com (Explanation of symbols REF 989-313-00).

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This image shows a full page of a document template designed for handwriting practice or general note-taking. It consists of approximately 20 evenly spaced horizontal dotted lines across the entire width of the page. The background is plain white, and there are no margins, headers, footers, or other markings present.

Notes

This image shows a full page of white paper with horizontal dotted lines. The lines are evenly spaced and run across the width of the page, providing a guide for handwriting practice. There are no margins, text, or other markings on the page.

[illegible]

Dentaurum

Germany | Benelux | España | France | Italia | Switzerland | Australia | Canada | USA
and in more than 130 countries worldwide.

DENTAURUM
QUALITY
WORLDWIDE
UNIQUE

- ➔ Informationen zu Produkten und Serviceleistungen finden Sie unter www.dentaurum.com
- ➔ For more information on our products and services, please visit www.dentaurum.com
- ➔ Vous trouverez toutes les informations sur nos produits et services sur www.dentaurum.com
- ➔ Descubra nuestros productos y servicios en www.dentaurum.com
- ➔ Informazioni su prodotti e servizi sono disponibili nel sito www.dentaurum.com

Stand der Information | Date of information | Mise à jour |
Fecha de la información | Data dell'informazione: 11/20

Änderungen vorbehalten | Subject to modifications | Sous réserve de modifications
Reservado el derecho de modificación | Con riserva di apportare modifiche

D
DENTAURUM