Venus® Bulk Fill

DE	Gebrauchsanweisung
GB	Instructions for use
FR	Mode d'emploi
ES	Instrucciones de uso
IT	Istruzioni per l'uso
PT	Instruções de utilização
NL	Gebruiksaanwijzing

Venus® Bulk Fill

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Venus® Bulk Fill

Product description
Venus Bulk Fill is a posterior flowable, light-curing, radiopaque nano-hybrid composite. It is used as a base in Class I and II restorations and can be placed in 4 mm increments. Venus Bulk Fill's flow on demand characteristics enable the material to adapt to the cavity walls. The universal shade allows curing of 4 mm layer thickness (base fill) with low shrinkage stress.
A universal restorative material is used in conjunction with Venus Bulk Fill as a facial/oral/occlusal enamel replacement. Venus Bulk Fill is compatible with methacrylate-based restorative materials designed for Class I and II occlusal restorations (e.g. Venus Diamond, Venus Pearl see Application).

Composition
Venus Bulk Fill is based on multifunctional methacrylate monomers (UDMA, EBA/MDA) and contains approximately 65% w/w and 38% vol inorganic fillers, such as Ba-Al-F-silicate glass, YbF₃, and SiO₂. The filler particle size is between 0.02 µm and 5 µm.

Color
Universal shade
Indicators
• Base in Class I and II direct restorations (up to 4 mm)
• Cavity lining under direct restorative materials in Class I and II

Contraindications
Do not use in cases of a known allergy to any of the constituents of Venus Bulk Fill. Application to the dentin less than 1 mm to the pulp

Side effects
Allergies to the product or its constituents may occur in isolated cases. If an allergy is suspected, request a list of ingredients from the manufacturer.

Warnings / Precautions
Avoid contact with skin or eyes. Use cause sensitization in case of skin contact. May irritate eyes. In case of contact with skin, wash skin with soap and water. In case of contact with eyes, rinse with plenty of water and seek medical attention.
PLTs are intended for single use. To avoid cross-contamination between patients, PLTs must be discarded after a single use!

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Manufacturer: Kulzer GmbH Leipzig Straße 2 63450 Hanau (Germany)
Made in Germany
Distributed in USA / Canada exclusively by: Kulzer, LLC 4315 South Lafayette Blvd. South Bend, IN 46614-2517 1-800-431-1785
Caution: Federal law restricts this device to sale by or on the order of a dentist.

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For deep cavities with only a minimal layer of dentin over the pulp or those which have already had a small area of pulp exposed, the pulp must be indirectly or directly capped locally using calcium hydroxide. The calcium hydroxide should be covered with a glass ionomer cement before Venus Bulk Fill is applied to the cavity. Prepared enamel and dentin surfaces on which Venus Bulk Fill will be adhesively tued must not come in contact with preparations containing eugenol, as these could inhibit optimal polymerization of the composite. Deviations from the instructions for use are the sole responsibility of the dentist. The product must be used solely in accordance with the instructions for use.

Application
NOTE: The shade selection of the final composite should be selected using the shade guide before inserting the dental dam to bias noticeably change color when dried.
1. Preparation, residual amalgam or restorative material from the cavity. Then rinse, clean and dry the cavity but do not desiccate the tooth structure.
2. Use a dental dam (e.g. Ivory® Rubber Dam) to isolate the treatment area from contamination. Refer to manufacturer's directions for placement.
3. Shape the proximal contacts with the aid of a matrix system. Using a matrix band with a wedge serves two purposes: compressing matrix band against the tooth structure and creating a seal against the tooth, compensating for the thickness of the matrix band, therefore helping create tight contact between the finished restoration and the adjacent tooth. Therefore, a thin matrix band is recommended. The matrix band should be burnished, or shaped, to achieve a contact facilitating its propagation into the cavity. The matrix band should also be shaped with a suitable instrument.
4. Apply the adhesive system (e.g., iBOND® Self Etch, iBOND® Total Etch) should be applied to the enamel and dentin according to the manufacturer's instructions before applying the restorability of the dental practitioner if another dentin/enamel adhesive product is utilized. Adhesive-treated cavities must not be contaminated with moisture, blood or saliva.
5. Application
a. Permanent teeth: Apply Venus Bulk Fill directly into the prepared cavities, starting at the deepest point and leaving the tip immersed in the material. Make sure that it flows slowly and evenly during application to prevent the formation of air bubbles. Leave the PLT immersed in the material and fill the cavity from the bottom to the top. Remove any air bubbles or excess filler with a probe or a microbrush before polymerisation. The distance to the occlusal margin of the cavity must be at least 2 mm.
b. Deciduous teeth: The cavity must be filled completely as described in 5a. Cover with water and eugenol-free dental composite. The thickness of the composite must not be less than 2 mm.
6. Alternative application of the use to Venus Bulk Fill in a thin layer as a liner on exposed dentin.
7. Polymerize Venus Bulk Fill for 20 seconds, on each of the restoration surface with a curing light (e.g., Translux®) for the table below (light output of 550 mW/cm²).
8. Venus Bulk Fill with a light curing unit (Translux®) for at least 20 seconds from each side of the restoration. When using plasma arc curing lights, the polymerization time can usually be reduced by 25%. When using LED (light emitting diode) curing lights, the polymerization times specified for curing with halogen units, provided direct contact between light source and the composite can be ensured. If the light intensity falls below 550 mW/cm², the curing time should be increased by 50%. It is recommended that polymerization times can vary significantly in intensity depending on the curing unit used. The polymerization times specified are therefore minimum times. In case of doubt, contact the lamp manufacturer for more information and check the functionality of the curing light in vitro before using it on patients.

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