FREEPRINT® MODEL PRO

DETAX
 Carl-Zeiss-Str. 4 · 76275 Ettlingen/Germany
 ☎ +49 7243/510-0 · 📠 +49 7243/510-100
 www.detax.com · post@detax.com

Made in
 Germany
 09/2023
 3

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EN Instructions For Use

FREEPRINT® MODEL PRO

DEUTSCH

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FREEPRINT® MODEL PRO

ENGLISH

Light-curing resin based on (meth)acrylate, for the generative fabrication of dental models for DLP printers with UV-LED 385 nm

Area of application

3D Print working and presentation models for the entire spectrum of models in dentistry

SUITABLE FOR THE FOLLOWING DLP PRINTERS/CLEANING/POST-EXPOSURE

see "Annex 1" (enclosed separately)

Processing

- The properties of the final product depend, among other things, on post-processing. Therefore it must be ensured that the light unit is in an orderly condition and that the models are completely cured (refer to the process description).
- Before use, the material should be shaken intensively and homogenized with a bottle roller.
- Processing temperature 23 °C ± 2 °C.

Safety Information

- Only to be used by trained specialists for the specified area of application.
- Product contains (meth)acrylates and phosphine oxides. The ingredients of **FREEPRINT® MODEL PRO** may cause allergic reactions in accordingly disposed persons. In such a case, do not use the product any further.
- Avoid direct contact with the liquid material and the components before post-curing. Irritating to airways, eyes and skin (sensitization possible).
- Wear personal protective equipment (protective gloves, goggles) when handling the uncured material.
- In case of contact with eyes, rinse immediately with plenty of water and seek medical advice.
- In case of contact with skin, wash immediately with plenty of soap and water.
- The processing instructions and precautionary measures must be observed.
- Refer to the relevant safety data sheet for hazard and safety information.

Notes

- DETAX does not accept liability for any damage caused by misuse.
- Always keep container tightly closed, carefully close immediately after each use.
- Product may cause allergic reactions.
- Read and understand the safety data sheet!

Storage

- FREEPRINT® MODEL PRO** is to be stored dry (at 15 °C - 28 °C) and protected from light. Minimal influence of light can already induce polymerisation.
- To protect against contamination, cover the material in the tray with the lid or a glass plate.

Disposal

Disposal of the contents/container must be carried out in accordance with the local/regional/national and international regulations.

Manufacturing process

Data preparation and fabrication of the support structure according to the instructions of the CAD software manufacturer

Construction process

Generation of a Print Job complying with machine and material parameters

Post-processing

After raising the platform, a drip time of approx. 10 minutes is recommended. If possible, post-processing should commence immediately following the construction process.


Cleaning

see "Annex 1, CLEANING EQUIPMENT"

Post-exposure

see "Annex 1, CURING LIGHT EQUIPMENT"

Storage:

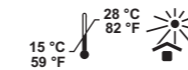Processing:

at 23 °C ± 2 °C

Ordering information:

FREEPRINT® MODEL PRO 385

grey
 1.000 g 04438

sand
 1.000 g 04439

caramel
 1.000 g 04440

FREEPRINT® MODEL 385

sand
 1.000 g 03778
 3 kg Eco Bag 04333
 5 kg Eco Bag 04321

ivory
 1.000 g 03780

grey
 1.000 g 03782

caramel
 1.000 g 02935

FREEPRINT® MODEL 2.0 385

sand
 1.000 g 02128
 3 kg Eco Bag 04331
 5 kg Eco Bag 04117

light-grey
 1.000 g 02099
 3 kg Eco Bag 04330
 5 kg Eco Bag 04107

grey
 1000 g 02177
 3 kg Eco Bag 04329
 5 kg Eco Bag 04106

caramel
 500 g 02844
 1.000 g 02850
 3 kg Eco Bag 04328
 5 kg Eco Bag 04015

white
 1.000 g 02148
 3 kg Eco Bag 04332
 5 kg Eco Bag 04118

FREEPRINT® MODEL T 385

blue
 1.000 g 02332
 3 kg Eco Bag 04334
 5 kg Eco Bag 04322

FREEPRINT® GINGIVA 385

500 g 02820
 1.000 g 02843