Instructions for use

myQuickmat Classico kit

sectional
matrix
systems

Index


EN - English	2	SE - Svenska	20
DE - Deutsch	4	DK - Dansk	22
FR - Français	6	PL - Polski	24
IT - Italiano	8	CZ - Čeština	26
ES - Español	10	TK - Türkçe	28
PT - Português	12	GR - ελληνικά	30
NL - Nederlands	14	JP - 日本語	32
NO - Norsk	16	CN - 中国	34
FI - Suomi	18	AR - العربية	36

Class II restoration using myQuickmat Classico kit

Sectional matrix system developed to offer optimal and gentle teeth separation using the NiTi myRing Classico in combination with the included interdental Maple Wood Wedges. The anatomically shaped, reusable, silicone Delta Tubes, ensure optimal matrix adaptation.


Instructions for use: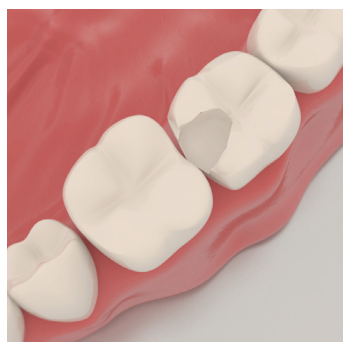


1. Prepare the cavity

Remove the carious lesion and prepare tooth with a conservative Class II preparation. Even if the contact was not totally broken the matrix can be placed.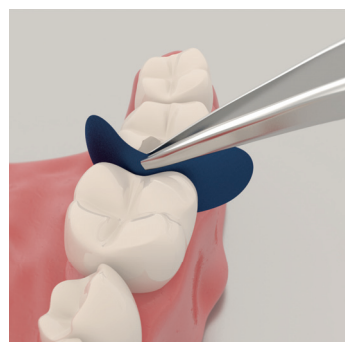


2. Place the matrix

Select an appropriate matrix band according to the tooth morphology and restoration type. Place the contoured sectional matrix with the concave edge towards the occlusal margin using your fingers or with tweezers.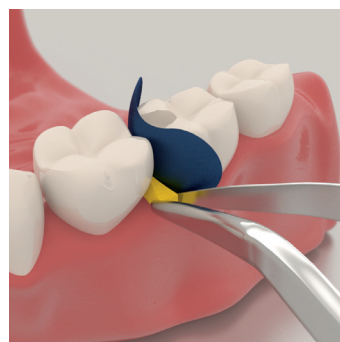


3. Wedging

Select the appropriate sized wooden wedge to close the gingival margin, and insert it firmly. When needed maintain a finger on the matrix band to avoid dislodging during wedging.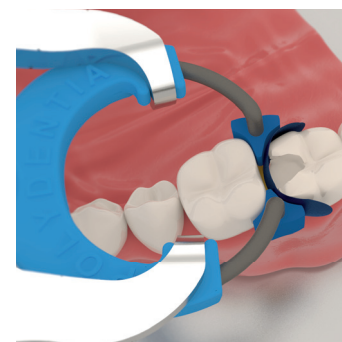


4. Place myRing Classico

Expand myRing Classico using Polydentia's myForceps or standard rubber dam forceps. Place the ring carefully and securely. Please refer to the Tips & Tricks section to see the different ways to position myRing Classico and ensure best teeth separation as well as matrix adaptation.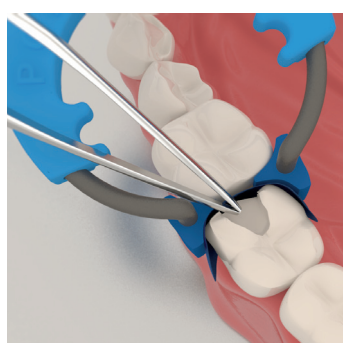


5. Burnish the matrix

For an excellent contact burnish the band against the adjacent tooth using a ball burnisher or tweezer.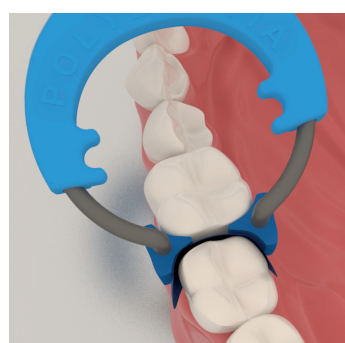

6. Restore the cavity

Restore the cavity according to the filling material manufacturer's instructions.

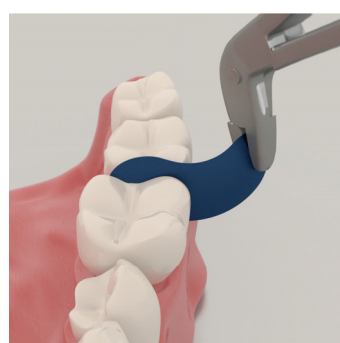7. Remove the sectional matrix system

Remove myRing Classico, the wedge and the matrix. Our suggestion for an easy matrix removal is to use the special Polydentia myQuickmatrix Forceps.

8. Finishing and polishing

With thin burs remove any small excess of material then polish the restoration according to the finishing and polishing instruments manufacturer's recommendations.


Recommendations:

- 1 - Place a rubber dam
- 2 - Read Cleaning, Disinfection and Sterilization Instructions at www.polydentia.com

Tips & Tricks using
myQuickmat Classico kit

Wedging


According to the clinical situation, the wooden wedge can be broken in order to better accommodate myRing Classico with Delta Tubes. This trick is very helpful in case of small teeth.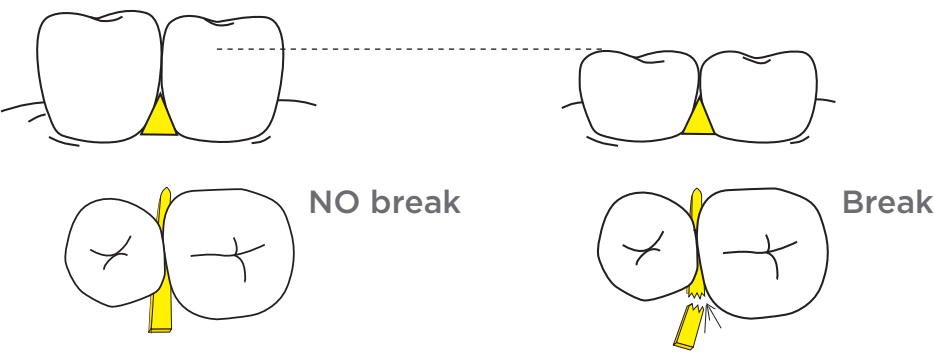

Delta Tubes placement options


Delta Tubes can be oriented according to the clinical situation to obtain the best matrix retention and adaptation to the tooth's anatomy. The most common cases are explained as follows (Case A, B). Different orientation possibilities can be combined to best adapt myRing Classico to each specific clinical situation. The use of Delta Tubes is not mandatory, myRing Classico can be placed also without Delta Tubes (Case C, D).

Case A, small cavities: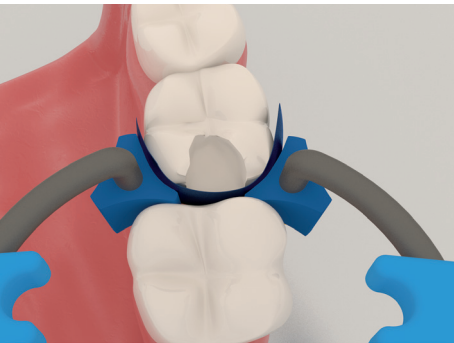


Delta Tubes edges allow a perfect adaptation between the teeth

Case B, broad cavities: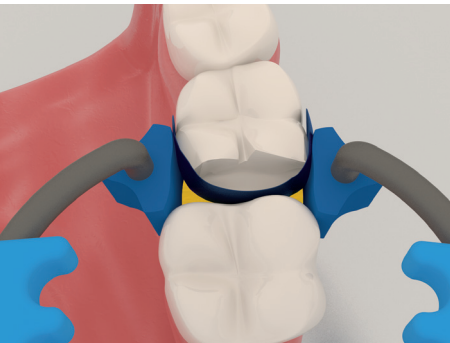

Delta Tubes flat surfaces provide support on the adjacent tooth, no matrix deformation.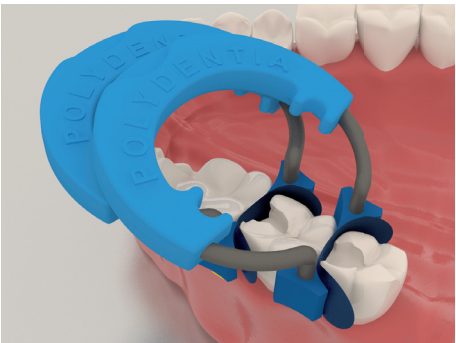


MOD and multiple tooth restorations:

MyQuickmat Classico rings are designed to be stacked over one another to perform a MOD and multiple tooth restorations.

1. Prepare the cavities as described in the instructions for use, step 1.
2. Place matrices and wedges on both mesial and distal surfaces of the tooth being restored
3. Position myRing Classico
4. Complete the restoration as described in the instructions for use, steps 5 to 8.

Case C and D, myRing Classico without Delta Tubes

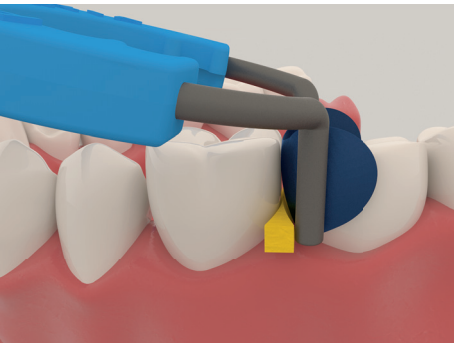myRing Classico tines placed near to the wood wedge

This placement technique is suitable for cases of a wide space in the interproximal area.

myRing Classico tines placed over the wood wedge

This placement technique is suitable if the interproximal space is tight or when the cavity is deep.

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