




neodisher® Alka 300



Alkaline detergent for the automated reprocessing of thermostable and thermolabile instruments

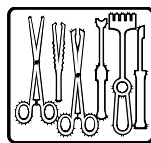

Liquid concentrate

Fields of application:

- Automated cleaning of instrument containers, surgical instruments and anaesthetic equipment, as well as containers and cages in animal facilities

Performance spectrum:

- Removes blood, protein residues and organic residues
- Suitable for instrument steel, chrome nickel steel, chromium-plated and nickel-plated materials, pure aluminium, aluminium-magnesium alloys and commonly used plastics
- Suitability tests should be carried out before use in the case of special instruments, e.g. optical instruments or anaesthetic and respiration materials, as well as anodised aluminium

Special properties:

- With active chlorine for intensive cleaning performance
- With oxidative decomposition action for stubborn organic residues
- Usable with any level of water hardness, but generally used with softened or deionised water

Application and dosage:

neodisher Alka 300 can be used in washer disinfectors. The dosing amount is 2 - 5 ml/l depending on the area of application and the degree of contamination.

Use suitable dosing devices.

General instructions for use:

- The neodisher Alka 300 solution has to be rinsed off completely (preferably with deionised water).
- Do not mix with other products.
- Rinse out dosing system including hose with water (preferably deionised) before changing product.
- For professional use only.
- Reprocessing should comply with all ordinances pursuant to the regulations on medical devices and should be performed with appropriate validated processes.
- Please observe the reprocessing recommendations of the medical device manufacturers according to the requirements of the DIN EN ISO 17664, the cleaning recommendations given by the manufacturer of the animal cages as well as the recommendations of the working group for cage processing" (AK KAB) in the current issue of the AK KAB brochure "Cage Processing in Animal Facilities properly done".
- The instructions given by the manufacturer of the washer disinfectors are to be observed.

Technical data:

Density	approx. 1.4 g/cm ³ (20 °C)
pH-range	11.1 - 11.9 (2 - 5 ml/l, determined in deionised water, 20 °C)
Viscosity	< 50 mPa s (concentrate, 20 °C)
Titration factor	1.04 (in accordance with neodisher titration method)

Ingredients:

Ingredients according to Regulation (EC) No 648/2004 on detergents:



neodisher® Alka 300

< 5 % chlorine-based bleaching agents

15 - 30 % phosphates

CE-mark: 

neodisher® Alka 300 complies with European guidelines for medical devices.

If a serious incident occurs with the product, report it to the manufacturer and the relevant national authority.

Storage information:

Always store at a temperature between -10 °C and 25 °C. Keep away from sunlight.

Usable for 1 year when stored as recommended.

For expiry date refer to the stamp mark on the label behind the hourglass symbol .

Hazard and precautionary statements:

For safety information see Safety Data Sheets.

These are available at www.drweigert.com under the category "Service/Downloads".

Dispose only when container is empty and closed.

For disposal of product residues, refer to the Safety Data Sheet.

MB 3103/3-2

Date of issue 04/2020

With the above information, which is appropriate to our current knowledge we describe our product regarding possible safety necessities, but we do not involve any quality description or promise certain properties.